

Part 1: Charging instructions

1. Connect the Micro-USB line to the charging case to let the charging case fully charged.
2. When you need to charge the hearing aid, put the machines into the charging case.
3. During charge period the indication light will show RED.
4. The indication light will not display once it is fully charged.
5. Charging time: 90min-2 hours
6. Operating time: 12 hours



Part 2: Operation Instructions

1. Before use, make sure the hearing amplifier are fully charged.
2. Taking out the hearing from the charging case.
3. Press up to 3 seconds to turn on when you hear twice "beep", the indication light will show WHITE.
4. While covering the microphone with your fingertip to avoid the high-pitched sound, place the hearing amplifier into ear.



5. Make sure you can use it comfortably, And the microphone is not blocked
6. To turn off the hearing amplifier, press up to 3 seconds when you hear twice "beep" the indication light will show BLUE.

Volume control:

Press the volume button to adjust the volume. There are 6 sound levels of volume, and you will hear a "beep" sound each time a level goes up. You will hear three times "beep" at the max volume. If you press again, it will go back to the lowest volume.

Instruction manual

Model:GM-305



Part 3:Specification	
Chanrging case working current:	≤100mA
Chanrging case working voltage:	3.7V
Chanrging case charging time:	90min
Chanrging aidcharging current:	≤50mA
Chanrging aidcharging voltage:	4.2V
Chanrging caseworking times (double machine) :	5-6times

Part 3:Specification	
Maximum OSPL90:	≤120dB±3dB
High frequency average OSPL90:	103dB±4dB
Average gain:	23dB±5dB
Equivalent input noise level:	≤35dB
Total harmonic distortion:	≤10%
Frequency response range:	300Hz~3500Hz
Rated power supply current consumption:	≤6mA
Work time:	30-36hours
Charging time:	90min-2hours