

User Instructions

PU Padded Corner Shower Stool M18319

Maximum User Weight:



General Safety Warning

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

Intended Use

The PU Padded Corner Shower Stool features a moulded polyurethane seat and is height adjustable. It is intended to be used in a shower to provide a stable, seated support, where a user is unable to stand for long periods (i.e. during showering).

Specification

Seat height Adjustable 510-660mm (20-26")
Maximum user weight 160kg (25st)

Height Adjustment

1. Place the seat upside down on floor/flat surface.
2. Adjust to the correct height by removing pin clips, sliding extension leg to required height and replace pin clip.
3. Ensure all three leg extensions are secured to the same height adjustment hole, to prevent stool becoming unstable.

Using the Shower Stool

Place in shower. Ensure all three legs are level and in contact with the shower floor. Sit centrally on the stool.

Safety Precautions

Do not exceed the maximum user weight limit.
Inspect at regular intervals for wear and damage.
Do not use if rubber feet are damaged.

Cleaning Instructions

Clean regularly, wiping down with mild detergent /disinfectant solution.



Manufactured for
Nottingham Rehab Supplies

Clinitron House,
Excelsior Road,
Ashby Park
Ashby de la Zouch,
Leicestershire, LE65 1JG
United Kingdom

Tel: 0845 120 4522

Int'l: +44 (0) 1530 418378

Fax: 0845 121 8112

Email: customerservice@nrs-uk.co.uk

This product is from a range of over 3500 daily living aids available from Nottingham Rehab Supplies. To view the full range visit www.nrs-uk.co.uk or request a catalogue from the above number.

