


Assembly Instruction for Toilet Safety Frame

①

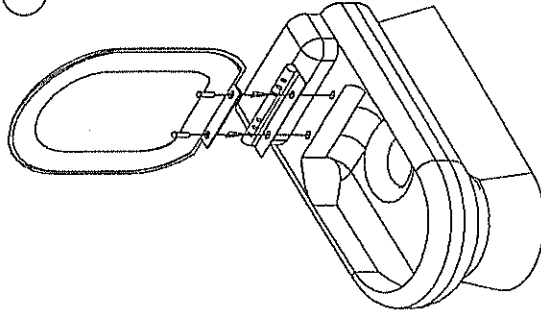


(a) Remove the toilet seat and cover from the toilet bowl.


(b) Install the bracket of the Toilet Safety Frame onto the rear-end surface of the toilet bowl. Replace the toilet seat and cover, please put the bolts of the toilet cover through the bracket hole and toilet bowl. Then replace original nuts and tighten.

②

Insert the L-shaped frame into the side hole of the bracket.

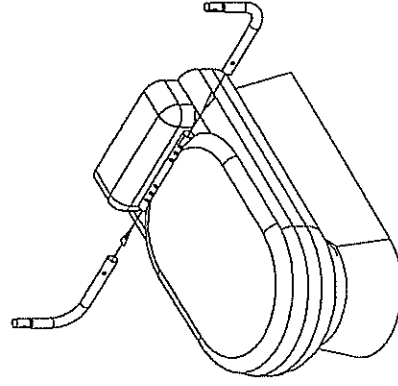


③

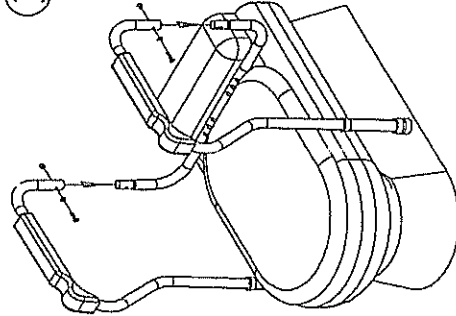

Install the side frame to the L-shaped frame and assure it have been joined tightly.

④

Sliding the leg up or down to the desired height by engaging the pin on the legs.

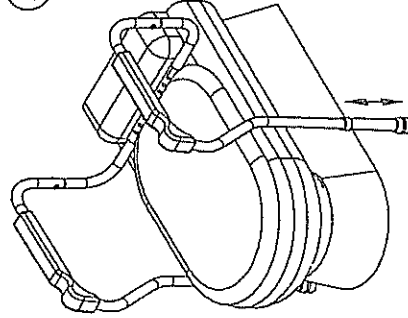

⚠WARNING

- Ensure the legs of Toilet Safety Frame are firmly contacted with the floor.
- Make sure ALL the pin protrudes fully through the holes.
- After Installation, please test for stability before use.
- The Toilet Safety Frame is designed to provide support and increase stability. It is not designed to support the total weight of an individual.