

# Alarm Pager PAG-03

The PAG-03 alarm pager with display will produce an alarm when signalled by pull cord transmitters, nurse call point and bed leaving alarm transmitters. **(Note: as part of our quality control during testing, the pager will have been set to English language and the batteries will be inserted)** Insert 2 x AA batteries into the battery holders observing the polarity markings. Before using the pager, the language announcement requires setting. To do this, remove the battery cover, you will see a switch marked "lock" "unlock". Slide the switch to unlock. You can select numeric announcement or a simple tone alarm. Press the language button repeatedly until you hear the voice announce number zero. Slide the switch back to lock position.



The pager can operate in vibrate or sound mode. To use the pager in sound only mode, slide the switch to the double note mark. For vibration only slide the switch down towards the double note with two lines.

When a signal comes in, the selected alarm type will operate, the LED on top of the pager will flash and the display back light will light showing the transmitter number.

The review button when pressed will display and announce previous alarm calls up to a maximum of six. As new calls are received the oldest stored call is automatically removed.

The volume button when pressed will adjust the alarm volume.

Low battery warning is provided by a flashing LED on top of the pager.