




PARNELL PREMIER BED RAIL
BRITISH STANDARD BS EN 1970:2000
REGISTERED DESIGN

MAXIMUM USER WEIGHT 112kg / 18st

ASSEMBLE THE BED RAIL AS SHOWN IN THE DIAGRAM • ADJUST THE HANDWHEEL
AND POSITION THE TWO FORKS UNDER THE BED FRAME, WITH THE BED RAIL
AT A POSITION TO SUIT THE OCCUPANT • TURN THE HANDWHEEL
CLOCKWISE UNTIL RESISTANCE IS FELT • CONTINUE TURNING
UNTIL SOME OF THE WEIGHT OF THE BED IS BORNE ON
THE BED RAIL • THE BED RAIL IS NOW READY FOR USE
• TO REMOVE THE BED RAIL REVERSE THE ABOVE PROCEDURE

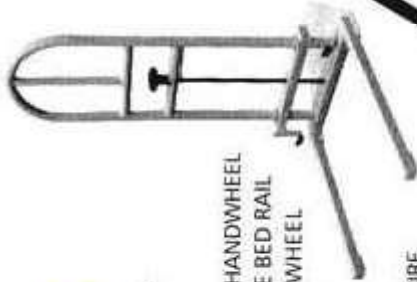

THE BED RAIL SHOULD BE PERIODICALLY CLEANED WITH NORMAL DOMESTIC
CLEANING MATERIALS, AVOID ACID, ALKALINE OR OTHER CLEANING
MATERIALS IN STRONG OR CONCENTRATED FORM AS THEY MAY
DAMAGE OR DISCOLOUR THE PRODUCT.

IT IS RECOMMENDED THE TWO HALVES OF THE BED RAIL
ARE SEPARATED FOR CARRYING.

PARNELL PRODUCTS LTD.

Tel: 0191 237 2096

Email:
enquiries@parnellproducts.com
www.parnellproducts.com



Bedside Independence for Life



The Parnell Premier Bed Rail range

is the ideal solution to help get in or out of bed.

- Designed for adjustable domestic beds also fits conventional domestic beds
- Simple three step assemble and fix to bed
- High strength and stability for maximum user reassurance and security
- Lightweight and flat pack for portability and storage
- No straps to interfere with bed changing

CE Independently tested to British Standard BS EN 1970:2000



For most beds and
general applications

Recommended max
user weight:


250lb / 18St / 115Kg



For increased bed height, greater
mattress thickness and heavier users

Recommended max
user weight:

350lb / 25St / 160Kg

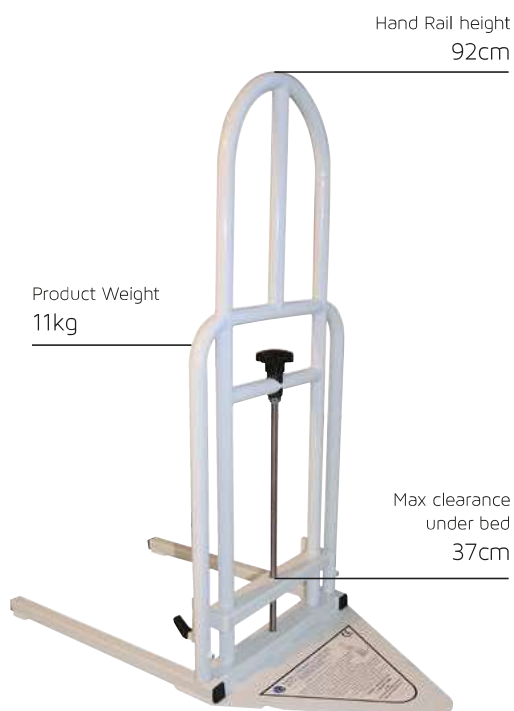

Standard sizes across all products

Hand Rail width 28cm
Floor Frame length 82cm

Floor Frame width 40cm
Min clearance under bed 3cm



Extra strong for bariatric use

Recommended max
user weight:

450lb / 32St / 200Kg



Registered Design



Independently tested to British
Standard BS EN 1970:2000

FLEXIBILITY

These unique bed rails adjust to almost any bed. Designed specifically for profiling beds they also fit conventional domestic beds. A wide range of adjustment allows the Parnell Bed rails to fit the maximum number of beds. They can be positioned on either side and at any point.

Can be used on: Profiling beds, Metal-framed beds, Wooden Beds, Divan Beds, Double or single beds.

EASE OF INSTALLATION AND CONVENIENT USE

Assembled and attached within minutes, the Parnell Bed Rail does not interfere with the mattress or bedding and can just as easily be removed and stored. No tools or special equipment are required and fitting can be carried out from a standing position without any awkward lifting of mattresses.

Simple three-step installation:

1. Place top section on base frame



2. Position where required



3. Turn black handle clockwise until tight and hooks are firmly fixed under the bed frame



To remove simply unwind the handle and slide away. The top section can be removed and placed flat on the floor frame for compact storage.



COMFORT

The Parnell Bed Rails do not fit under the mattress and so do not affect the comfort of the bed.

The ease of positioning and the range of products mean that finding the optimum position for each individual is quickly achieved and the frame design always provides a comfortable hand hold.



QUALITY

With a British design and independently tested to EN 1970:2000 each component and material has been carefully selected and assessed for strength and durability. The high quality finish includes premium powder coating and stainless steel rods and will provide years of trouble free use.

A full 12-month guarantee is provided.



REASSURANCE

The unique design of the Parnell Bed Rail provides exceptional strength and rigidity giving the user unparalleled reassurance. As it is attached firmly between the bed and the floor, it does not rely on user weight and provides proper support at all times.

Rigorous testing to EN 1970:2000 has demonstrated minimal flexing under significant load cycles so you can be sure the Parnell Bed Rail will not let you down. It also means that there are no head or finger entrapment points and only smooth surfaces, ensuring maximum safety and hygiene.

The three Parnell Premier Bed Rails in this brochure are the only ones known to be CE Marked and tested to a recognised British Standard by a respected independent Test House. This is your assurance they meet their safety and legal requirements.

PARNELL PREMIER BED RAIL RANGE
BS EN 1970:2000 All IP Rights Protected
 Registered design for adjustable domestic beds (and conventional domestic beds)

Please note that all dimensions and weights stated are approximate. Parnell Products Ltd reserve the right to alter specifications without notice due to continuous product development.

For further information and the latest prices please contact:
 01670 829700 • mail@parnellproducts.com • www.parnellproducts.com