

### Product Range

P23117 Duo Walking Trolley - Turquoise Blue

P23129 Duo Walking Trolley - Pebble Grey

### Product Description & Intended Purpose

The Duo Walking Trolley is intended to be used as a household trolley to transfer items around the home for those with reduced mobility. The product is height adjustable and has been designed to allow a natural walking gait.

### Product Features

- Dual action sliders allow the trolley to slide when pushed forward but provide grip and stability if the user leans on the frame.
- Lightweight powder coated steel frame.
- Design allows for a natural walking gait.
- Removable trays for easy cleaning.
- Height adjustable to suit a range of users.





### Technical Specification

#### Safe Working

**Load:** 20st  
(127kg)

**Tray Capacity:**  
5kg (11lb)

**Product Weight:**  
6kg (13¼lb)

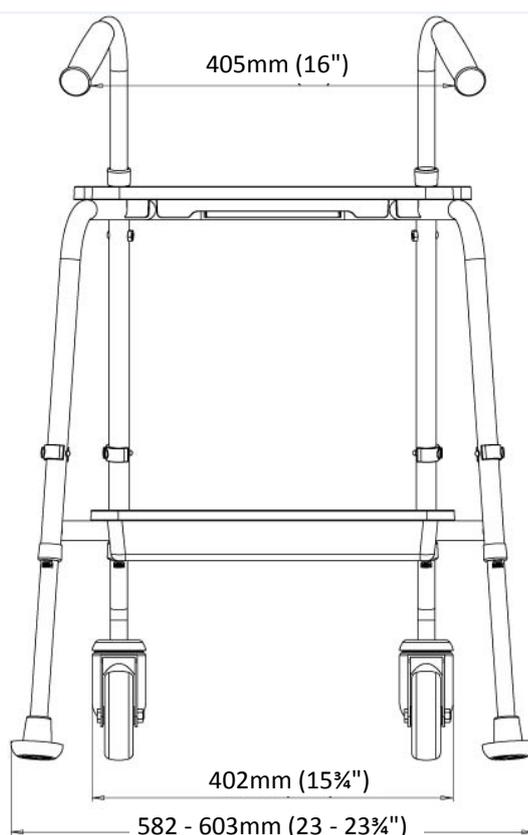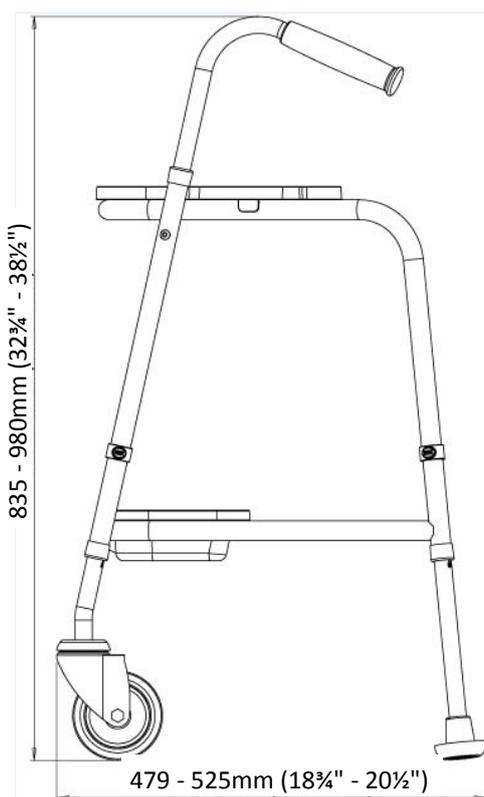

# PTDS048

## Duo Walking Trolley Range

### Installation Information: *(Note: for detailed information refer to User Instructions)*

- This product requires assembly before use, it is supplied in 'flat pack' form.
- All tools and fixings are supplied with the product. No specialist skills are required.
- Assembly time is approximately 10 minutes.
- It can be adjusted for height to suit individual users.

### Safety Precautions

- Do not use the trolley to assist when attempting to stand from a seated position.
- Do not overload the trolley trays, maximum loading 5kg. If possible it is advisable to position the items centrally on the trays.

### Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water.
- If required the trays can be removed for cleaning, simply unclip from the frame.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- Remove any build up of dirt or debris from around the wheels and sliders.

### Maintenance

- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Regularly check the condition of the sliders and that they operate correctly.
- Check the wheels can rotate freely.

### Spares / Accessories

- N43188 - Upper Tray
- N43199 - Lower Tray
- N43229 - Replacement Slider Ferrules - Pair
- N43218 - 'e-clips' - Pack of 4
- F17628— Dycem non-slip mat - Red
- F17639— Dycem non-slip mat—Blue

### FAQ's

- How do I measure the correct height for the handles?

*Stand with your feet flat on the floor, in your usual footwear, with your arms held loosely at your sides and measure the height from the floor to your wrist.*

### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK