

USER INSTRUCTIONS

A SERIES 4 WHEEL ROLLATOR

P11255

Rollator - Red

P11267

Rollator - Blue

P11279

Rollator - Silver

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

The A-Series Rollator is a lightweight, aluminium rollator designed to provide support and assistance to those unable to, or who lack the confidence to, walk without some level of guidance. This rollator is intended for indoor and outdoor use.

2. SAFETY

DO NOT exceed the maximum user weight 21st (135kg).

MAKE SURE the rollator is locked in the open position before use.

ONLY use as a walking aid in the forward direction.

DO NOT use on stairs, escalators or steep hills with gradients greater than 1 in 8.

MAKE SURE both brakes are locked before using the seat and that the backrest is fitted.

DO NOT move the rollator when the seat is occupied.

DO NOT sit on the seat when the rollator is on an incline.

DO NOT hang items from the handles as this may affect the stability of the rollator.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

3.1 Contents:

1. Rollator main frame x 1
2. Handles x 2
3. Backrest x 1

Maximum User Weight



Figure 1



Figure 2



Figure 3



4. ASSEMBLY/INSTALLATION

4.1 Unfold the Rollator:

Push down on the black bar under the bag until the cross bars are horizontal and the wheels are fully extended.

4.2 Insert the handles:

Slot the handle tube inside the main frame.

Adjust the height such that the handles are in line with your wrists when you stand upright with your arms relaxed by your sides.

Align the holes in the tubes and insert the hand wheel, figure 2.

Check the hand wheel is on the outside of the frame.
Tighten the hand wheel to secure.
Ensure both handles are set to the same height.
Check the brake cable is attached:
Clip the brake cable onto the main rollator frame, figure 3.

4.3 Attach the backrest:

Press and hold the black button on the backrest housing.
Slide the backrest tube into the housing slot, figure 4.
Release the button to secure.

5. IN USE

5.1 To operate the brakes:

Squeeze the brake lever up to apply the brakes.
Let go of the lever to release the brakes.

5.2 To lock the brakes:

Push the brake lever down until it clicks in position.

5.3 To unlock the brakes:

Pull the brake lever up. An audible click will be heard as the brake is released.

5.4 To fold the rollator:

Pull the backrest towards the handle.
Lift the seat up.
Pull the black cord up to draw the wheels together.



6. MAINTENANCE

Check all fasteners (screws, nuts, bolts etc.) are correctly located and securely fitted.

Check the wheels can rotate freely.

Ensure the brakes are functioning correctly. When squeezed or locked the two rear wheels should not be able to rotate.

Check for general wear or damage. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

7. CLEANING INSTRUCTIONS.

Ensure all dirt and debris is removed from around the wheels, axle and brakes.

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products prior to use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; fair wear & tear, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

If you have any problems with our products, please contact us at the address below.

If you have experienced an injury, contact us but feel free to contact the Medical and Healthcare products Regulatory Agency at:

devices.regulatory@mhra.gov.uk



NRS HEALTHCARE Sherwood House,
Cartwright Way, Forest Business Park, Bardon
Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111



NRS HEALTHCARE 4C Altona Road, Blaris
Industrial Estate, Lisburn, Co Antrim,
Northern Ireland BT27 5QB

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

www.healthcarepro.co.uk or www.completecareshop.co.uk