

User Instructions P11942 NRS TOILET FRAME & SEAT HEIGHT.ADJ 45-60CM

General Safety Warning

Please ensure these instructions are fully read and understood before installation/use. Retain these instructions in a safe place for future reference.

Maximum User Weight



1.0 Intended Use

The Height Adjustable Toilet Frame with seat and lid is designed to assist those who have difficulty in lowering onto or rising from a standard toilet. It should be used over a toilet using the splash guard chute supplied.

2.0 Instructions for use

1 – Fit the splash guard chute by raising the toilet lid and seat and placing onto the wire side brackets

2.1 Seat height Adjustment

2 - Adjust to the required height by removing each 'e-clip' in turn.

3 - Slide each leg up or down to the required height. Align the holes and reinsert the 'e-clip'.

4 - Ensure that all four legs are set at the same height to prevent rocking.

5 - The device may then be placed over the toilet and is ready for use.

3.0 Safety Precautions

1. Check the feet monthly for signs of wear.
2. Check 'e-clips' every 2 months.
3. Do not exceed the maximum user weight.
4. Do not sit on the lid



4.0 Cleaning Instructions

- Clean regularly using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could damage surfaces and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution and rinse thoroughly.
- Allow the product to dry prior to use/storage.

5.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare,

Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville Leicestershire, LE67 1 UB, United Kingdom

Tel. Head Office: 0330 100 00 98

Tel. Customer Services: 0845 120 4522

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk