

PRODUCT INSTRUCTION SHEET

REFERENCE ROTHLEY PORTABLE TOILET FRAME (160TF)

DESCRIPTION

An adj. metal stand and support frame for use with a port-potty. Height adjustable and can be disassembled and flat-packed for storage or transportation. Frame has four non-slip feet fitted with 2 polyurethane moulded armrests.

TECHNICAL SPECIFICATIONS

Height of Platform 55 – 155mm (2” – 6”)
Height of side rail 725mm (28.5”)
Width: 21”
Width between arms: 16”
Depth : 17”
Weight 5.7kg



USAGE REQUIREMENTS

1. Do not exceed maximum user weight of 165kg (25 St).
2. Use on level surfaces only.
3. Adjust to the correct height by removing pin clips, sliding extension leg to the required position and replace pin clip.
4. Ensure all 4 leg extensions are secured to the same adjustment hole to ensure even height.
5. At all times, ensure there is no gap between the horseshoe clip and frame.
6. Place Porta-potty into the frame, making sure it rests in a level position on the 4 corner brackets.
7. Clean regularly, wiping down with mild detergent/disinfectant solution. Do not use bleach based cleaning products on the PU armrest's this may lead to discoloration?