Description	REF
Uni-Frame Folding Shower Chair	081305358
Uni-Frame Folding Toilet Rail	081124551
Uni-Frame Folding Toilet Frame	081124619
Uni-Frame Folding Commode	081305606
Screw-Down Feet	081124627

Intended uses

- To help increase independence when using the shower or toilet, particularly by compensating for impaired movement.

Assembly and Use

Basic Frame

Adjusting the height (Figure 1):

With the frame lying on the floor, remove the pin-clips (A), which release the leg sleeves (B) and the legs (C). Adjust to the desired height, align the holes, and replace the pin-clips. Be sure all of the legs are adjusted to the same height.

Attaching the components (Figure 4, showing the Shower Chair):

The frame can be used with a variety of rails and seats. Attach the components as follows:

- Clip the back of the rail or seat (D) onto the back bar of the frame (G) and push firmly into place.
- Position the front pegs on the rail or seat into the holes (F).
- Position the back pad (E) in the centre of the back of the upper frame and push it firmly into place.

Folding the frame (Figures 2 and 3):

To fold the frame, pull the front pegs out of the holes and swivel the rail or seat down. Then fold one set of legs across the front of the frame and the other set across the back.

Using the optional screw-down feet:

If the frame is to be used in only one place, it can be attached permanently to the floor using the screw-down feet (081124627). To use the screw-down feet, remove the leg sleeves with the slip-resistant feet and attach the screw-down feet in their place. Attachment to the floor should be done by a qualified tradesman.

Uni-Frame Folding Shower Chair (081305358) - see Figures 4 and 5

- Attach the shower chair (D) to the frame as described in Attaching the components.

Uni-Frame Folding Toilet Rail (081124551) - see Figures 6 and 7

- Attach the surround rail (H) to the frame as described in Attaching the components.

Uni-Frame Folding Toilet Frame (081124619) - see Figures 8 and 9

- Attach the rail (J) to the frame as described in Attaching the components.
- Position the seat (K) in the centre of the frame and apply pressure to the seat to clip it securely into place.

Uni-Frame Folding Commode (081305606) - see Figures 10 and 11

- Attach the rail (L) to the frame as described in Attaching the components.
- Position the commode pot (P) so it rests on the bars of the commode rail.
- Remove the commode pot lid (N).
- Rest the commode seat (O) on the rail so the attachment pins fit inside the bars on the commode rail.
- The back panel (M) can be used as a backrest while the commode is being used and as a lid.

Care and Cleaning

All parts can be wiped using a brush or cloth and a solution of mild detergent or household disinfectant.

⚠ Cautions

- The Uni-Frame Folding Shower Chair (081305358) is not to be used to support more than 120 kg / 264 lb / 183/4 st; the Uni-Frame Folding Toilet Frame (081124619) is not to be used to support more than 127 kg / 279 lb / 20 st.
- Be sure the frame and accessories are securely in place before putting weight on them.

Uni-Frame Folding Shower Chair (081305358)

- Use of a slip-resistant bath mat is strongly recommended.
- Cover the seat with a towel to help reduce

Serious events experienced from using this product should be reported immediately to Performance Health and to the local Competent Authority.

Symbols Glossary located at <http://www.performancehealth.com/medical-device-glossary>.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy.

Please retain these instructions for future reference.

Performance Health International Ltd.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK
Tel: +44 1623 448 706
www.performancehealth.co.uk

Performance Health France
13 rue André Pingat CS 10045
51 724 Reims Cedex France

Distributed in the United States By:
Performance Health Supply, LLC
W68 N158 Evergreen Blvd
Cedarburg, WI 53012, USA
+ 1800-323-5547
www.performancehealth.com

Distributed in Australia By:
Ausmedic Australia, Trading as
Performance Health ANZ
Unit 3, 3 Basalt Road
Pemulwuy NSW, 2145, Australia
1300 473 422
www.performhealth.com.au

Distribué en France par:
Performance Health France
13 rue André Pingat CS 10045
51 724 Reims Cedex France
00(33) 03 10 00 79 30
www.performancehealth.fr

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