

USER INSTRUCTIONS

SHOWER CHAIR RANGE

F19054	Fixed Height	G31199	Fixed Height, Mobile
G31206	Height Adjustable	L64488	Height Adjustable, Mobile

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

To provide stable seating for users with reduced mobility who are unable to stand whilst showering. The static models (F19054 & G31199) are suitable for use on a shower tray whilst the mobile models (G31199 & L64488) are ideal for wet rooms.



G31206 Height Adjustable



L64488 Height Adjustable Mobile

2. SAFETY

Warning: The maximum user weight must not be more than 165 kg (26 st).

Always perform a risk assessment prior to use, including safe moving and handling guidelines.

Do not use if castors or ferrules are damaged.

Do not attempt to climb or stand on the product.

Models F19054 & G31199

Ensure all four legs are securely fixed at the same height prior to use.

Models G31206 & L64488

Lock all four castors before transferring the user to and from the chair or when leaving the user.

Plan the route to avoid obstacles prior to use. Only use on level surfaces and take care over door thresholds.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. ASSEMBLY/INSTALLATION

Product arrives fully assembled.

5. IN USE

5.1 To Adjust the Height (G31206 & L64488)

Remove 'e-clip'.

Slide the leg to the desired height and align the holes.

Re-insert the 'e-clip', check the pin is fully located.

Ensure all four legs are secured at the same height to prevent rocking or instability.

6. MAINTENANCE

Check that all fixing clips are inserted correctly and there is no excessive movement.

F19054 & G31199 - Check the condition of the ferrules.

G31206 & L64488 - Check the castors are securely fitted and are free to rotate. Remove any build-up of dirt or debris.

Periodically inspect the product for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

7. CLEANING INSTRUCTIONS

Clean regularly using a general purpose neutral detergent and warm water.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

The shower chair is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 80°C.

Always allow product to fully dry prior to use/storage.

8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.	
Tel: Customer Services: +44 (0)345 121 8111	Email: customerservice@nrshealthcare.co.uk
NRS Healthcare is a leading supplier of community equipment and daily living aids. Please visit one of our websites to view our full range of products: www.healthcarepro.co.uk or www.completecareshop.co.uk	