

User Instructions

Corner Shower Chair

H84059

Maximum User Weight



General Safety Warning

Please ensure these instructions are fully read and understood before use. Retain in a safe place for future reference.

Intended Use

The Corner Shower Chair is an adjustable chair designed to sit in the corner of the shower, taking up minimal space.

Technical Specifications

Seat: Polypropylene with drainage slots

Frame: Tubular steel with "Warm touch" high performance plastic coating

Dimensions:

Overall Width: 571mm (22½")
Overall Height: 381mm (15")
Adj. Height: 495-647mm (19½"-25½")
Weight: 6kg (13lbs)

Fitting Instructions:

Check the shower tray can take the recommended weight before use.
Ensure the chair is level before being used
When adjusting the legs, ensure there is no gap between the "E" clip and the frame.
Ensure that all the legs are fixed in the same position to prevent rocking.
Periodically check the legs for excess movement.

Important:

Do not place weight on the seat when removed from the frame. It is designed to be supported by the frame, and only in this position will it conform to the maximum weight limit stated.



Cleaning Instructions

Clean regularly, use a general purpose, neutral detergent and warm water. Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.

To disinfect, wipe with a mild chloride disinfectant (solution of 1000 parts per million).
Autoclave at 80 °C

LOCAL AUTHORITY AND HEALTH SERVICE EMPLOYEES:

These instructions must be left with the unit for the User to refer to.

Manufactured for

NOTTINGHAM REHAB SUPPLIES

Clinitron House, Excelsior Road,
Ashby Park, Ashby-de-la-Zouch,
Leicestershire, LE65 1JG, United Kingdom

Tel: 0845 120 4522

Fax: 0845 121 8112

Int'l: +44(0) 1530 418378

Email: customerservice@nrs-uk.co.uk