

## Product Technical Data Sheet 018

### Wall Mounted Shower Seat

#### Product Range

M48787	Wall Mounted Shower Seat with Legs
M53552	Wall Mounted Shower Seat with Legs & Padded Seat

#### Product Description & Intended Purpose

To provide seated support for clients with reduced mobility who are unable to stand whilst showering. It also enables for safer side transfers. Folds flat against the wall for storage and to be non obstructive to other users.

#### Product Features

- Powder coated, height adjustable steel frame.
- Rubber ferrules for added grip and stability.
- Arm rests available.
- Anti slip ridges on seat or seat pad and arm rests.
- Moulded plastic seat complete with drainage holes.



#### Technical Specification

##### Dimensions (mm)

- |                         |                                 |
|-------------------------|---------------------------------|
| • Total Width: 490      | • Seat Height: M48787 455 — 585 |
| • Projection: 420       | M53552 475 — 605                |
| • Folded Projection: 95 | • Seat Width: 465               |
|                         | • Seat Depth: 380               |

**Maximum User Weight : 25st (160kg)**

#### Safety Precautions

- Do not install in fibreglass / acrylic shower enclosures with insufficient stiffness and strength.
- Ensure the legs are securely located at the desired height position before use.
- Do not attempt to stand on the seat.
- Do not exceed the maximum user weight, 25st (160kg).

# Product Technical Data Sheet 018

## Wall Mounted Shower Seat with Legs

### Installation Information: *(Note: for detailed information refer to User Instructions)*

- The product should be installed by a qualified and competent person.
- Check the structure of the wall it is to be fixed to, avoid electrical wiring or plumbing.
- To adjust the height; remove the 'e-clips' slide the legs to the desired height and re-insert the 'e-clips'. Choose the correct height for the client and position the seat accordingly.
- Secure the seat to the wall using appropriate fixings. It is recommended to use non-rusting fixings and to apply sealant around wall contact points to avoid water ingress.
- Check the seat can be raised and lowered without interference, and that the feet are securely in contact with the shower base.
- Check seat is firm and secure.

### Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water.
- To disinfect use a mild bleach solution and rinse thoroughly with fresh water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Always allow the product to fully dry prior to use or storage.

### Care & Maintenance

- Check that all fixing clips are securely fastened and there is no excessive movement.
- Check the condition of the ferrules.
- Periodically check the product for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of service.

### FAQ's

- Can the shower seat be used over a bath?

*It is not recommended for use over the bath, the space required for the wall fixing is not sufficient and the seat would be unstable.*

### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardonia Hill, Coalville, Leicestershire. LE67 1UB UK