

User Instructions

Swedish Bath Rail

M18515 – White

M20133 – Chrome

General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

The Swedish Bath Rail is designed to help steady the user while transferring into or out of the bath. It clamps to the side of a standard bath and can be fixed to the floor for added stability and support.

2.0 Safety Precautions

WARNING: This product may not fit securely to certain types of unusually shaped baths, including roll tops. Please check suitability before installation/ permanently fitting.

Do not exceed the maximum user weight 23st, 150kg.

Not suitable for use by children.

Check the clamps are secure and the hand wheels are tight prior to use.

3.0 Unpacking

Remove packaging and check all parts carefully before assembling the product. Do not assemble if any parts are damaged.

4.0 Assembly

Choose the desired position and orientation of the bath rail. It is advisable to have the handle pointing away from the tap end of the bath.

Maximum User Weight



Clean the bath wall thoroughly in the area where the rail is to be fitted.

Slide the clamps onto the handrail tubes such that the orientation of the handle is maintained.

Slide the handrail tubes inside the base

5.0 Installation

Adjust the height of the frame by sliding the handrail tubes inside the base tubes. Secure at the required height by aligning the holes and inserting an 'e-clip',

Ensure the base plate is sitting flat on the floor.

Slide the clamps such that they sit over the edge of the bath. Secure in place by tightening the hand wheel (H2) on both clamps.

Fit the rail to the bath by adjusting the clamps to size and tightening in position using the hand wheel (H1) on both clamps.

Take care not to overtighten the clamps when fitting to fibreglass and plastic baths as clamps could crack the bath.

For added stability, secure the bath rail to the floor using appropriate fixings.

6.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.


Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow product to dry thoroughly prior to use.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business park, Bardon Hill,
Coalville, Leicestershire, LE67 1UB, UK

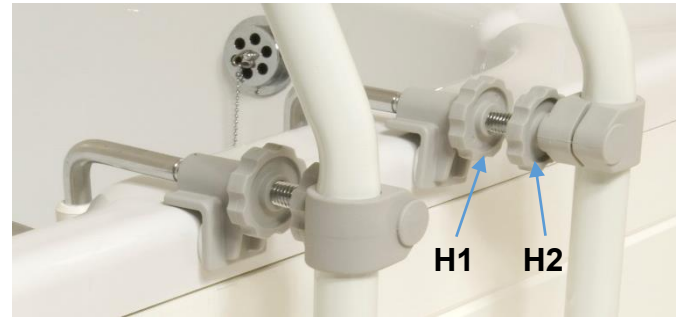

7.0 Maintenance

Check the clamps are secure and the rail has no excessive movement.

Periodically check the rail for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of service.

8.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website: