

NRS EasyFit Plus Bed Rail

The NRS EasyFit Plus Bed Rail provides stable support when getting in & out of bed



Designed & developed by NRS Healthcare

The EasyFit Plus Bed Rail provides stable support to aid user transfer in and out of bed, move from a lying to sitting position and assist with repositioning.

With a sleek, sophisticated design the EasyFit Plus Bed Rail is a complimentary addition to the bedroom environment promoting freedom and independence.

The EasyFit Plus Bed Rail has been designed to work with most beds, including slatted framed beds, solid base divans, sprung divans and non-height adjustable profiling beds. It can also be fitted to height adjustable beds where the height adjustment function has been disabled.

The ergonomic handle offers multiple grip options allowing users to find the most comfortable position.

Product features



- Designed to work on most beds including slatted framed beds and solid divans
- Provides stable support when getting in and out of bed
- Quick and easy to install – no tools or straps required
- NEW for 2020 - Easy use clamps slide quickly into position under the bed and secure easily
- Wide, low base plate and splayed legs for increased stability
- Strong construction – weight capacity 25st / 160kg
- Two models available: Standard P63991 with handrail height of 820mm and Tall P64004 with handrail height of 920mm
- Stored flat when not in use



UK Patent applied for
UK, European & Chinese Design
Registered

Assembled in 4 easy steps, with no tools or straps required, thanks to the unique sliding clamp design



1. The EasyFit Plus handrail assembles to the base without tools. Simply slot the handrail onto the base and position under the bed.



2. Slide the blue clamp levers outwards to unlock the clamps, then slide the clamps up the handrail until they contact the bottom of the bed frame.



3. Release the blue clamp lever to lock the clamps in place.




4. Turn the adjustment lever a few turns to firmly secure the rail to the bed frame. The EasyFit Plus Bed Rail is now ready to use.

EasyFit Plus Bed Rail Specifications

EasyFit Plus (Tall)
Product Code: P64004

EasyFit Plus (Standard)
Product Code: P63991

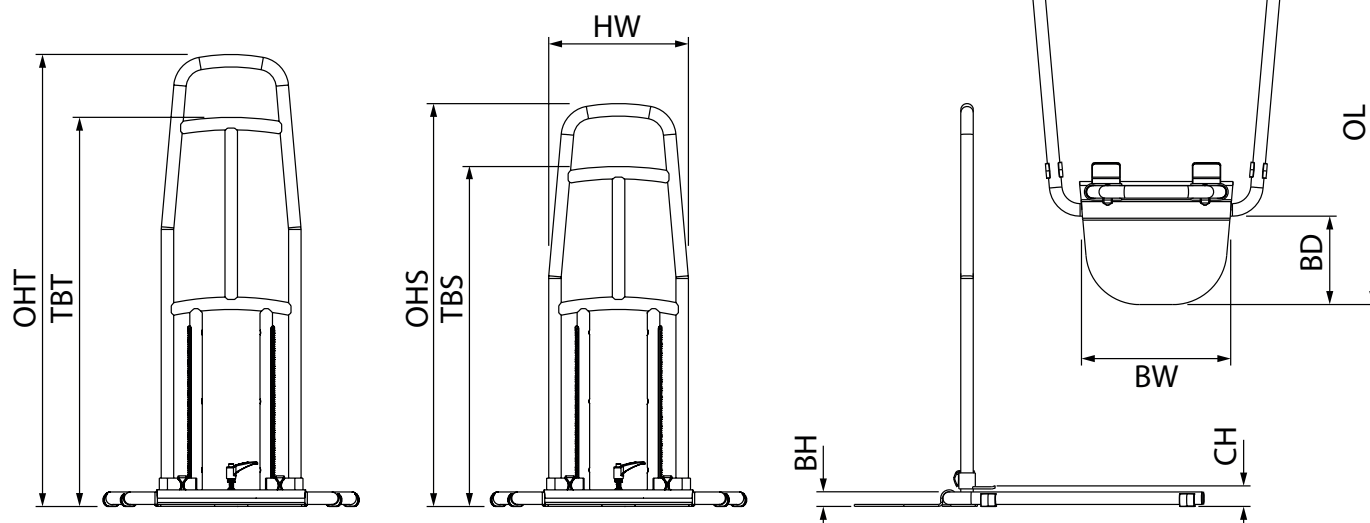

Symbol	Description	Dimensions
OW	Overall Width	522mm (20½")
OL	Overall Length	712mm (28")
OHS	Overall Height (Standard)	820mm (32¼")
OHT	Overall Height (Tall)	920mm (36¼")
HW	Handrail Width	285mm (11¼")
TBS	Top Bar Height (Standard)	692mm (27¼")

TBT	Top Bar Height (Tall)	792mm (31¼")
CH	Clamp Height (Min)	30mm (1¼")
CH	Clamp Height (Max)	330mm (13")
BW	Base Width	300mm (11¾")
BD	Base Depth	180mm (7")
BH	Base Tube Height	29mm (1⅛")
-	Max User Weight	160kg (25st)

Also available from NRS Healthcare NRS EasyFit Bed Guard: P42434

The NRS EasyFit Bed guard is designed to prevent the user from rolling out of bed.

Width adjustable base designed to fit a variety of bed sizes.

Guard extends or retracts to enable easier access to and from the bed.

No specialist tools or skills required for assembly.

Suitable for divan or slatted beds, from single to king size.

Guard can be detached from base to facilitate bed making.



For more information or to place an order, please contact your local Regional Sales Manager or the NRS Healthcare Customer Service Team:

Telephone: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

www.nrshealthcare.co.uk

For international enquiries, please telephone: +44 (0)1530 232 292

or email: exportsales@nrshealthcare.co.uk

