

USER INSTRUCTIONS

MOWBRAY TOILET SEAT & FRAME RANGE

M11089	Mowbray Width Adjustable (Free Standing)
M92739	Mowbray Toilet Seat & Frame 510 (Free Standing)

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

The Mowbray toilet frame and seat range is designed to assist those who have difficulty in lowering onto or rising from a standard toilet. The plastic moulded seat offers support in a natural position and the frame is height and width* adjustable.

(*Only available with M11089 – Mowbray width adjustable version).



2. SPECIFICATION

1. One piece moulded plastic seat.
2. White epoxy coated frame.
3. Non-slip ferrules (free standing versions)

3. SAFETY

WARNING: The maximum user weight must not be more than 190Kg (30st).

DO NOT: attempt to stand or climb on the product.

MAKE SURE: all legs are set at equal height to avoid rocking.

4. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

5. FITTING INSTRUCTIONS

1. Remove the moulded seat from its polythene bag and snap into place on the cross members of the frame.
2. Ensure the seat is firmly in place and secure at both back and front.
3. Adjust to the required height by removing each 'E' clip in turn.
4. Slide each leg up or down to the required height. Align the holes in and replace 'E' clip.
5. The back legs can be set at a slightly higher position than the front to help with ease of transfer. (Height difference should not be more than 2 holes).
6. Ensure that the two front legs are set at the same height and the two back legs at the same height position in order to prevent rocking.
7. Mowbray Width Adjustable only. (M11089). To adjust the width, remove 'E' clip either side of the seat and slide tubes to obtain the desired width (ensuring equivalent holes are used on each side of the seat so that the frame remains square. Replace 'E' clips.
8. Place the Mowbray over the toilet with the cutaway of the seat at the front and the integral splash guard placed centrally over the toilet bowl.
9. When not in use Mowbray's can be stacked with the seat in situ. It is recommended that no more than five frames are stacked together for long term storage.

6. MAINTENANCE

- Check the ferrules on the freestanding
- versions monthly for signs of wear
- Check floor fixing screws weekly
- Check E clips every 2 months

7. CLEANING INSTRUCTIONS.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

Regularly clean with a non-abrasive cleaner.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

This product is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 70°C.

Allow product to fully dry prior to use/storage.

8. ACCESSORIES & SPARE PARTS

F11572: Floor Fixing Kit (Set of 4)

F17938: Ferrule

G30615: Standard Seat

9. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal & plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

Household Waste and Recycling Centres.

10. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk