

Product Range

L22056	Standard Fixed Height
M66119	Height Adjustable
M99448	Wide Fixed Height

Product Description & Intended Purpose

Mobile commode for use over the toilet or in another area

Enables you to use your toilet by wheeling the commode over it, you will benefit from fewer transfers, making it a practical and portable toileting solution.



Product Features

- Portable and strong toileting solution
- Detachable arms for easier transfers
- Braked castors for heightened safety
- Pram style handles give carers easy manoeuvrability

Technical Specification:

- **Depth of Seat:** 490mm,
- **Seat Height:** 470mm (L22056, M99448), 460—535mm (M66119)
- **Width Between Arms:** 450mm (L22056, M66119), 480mm (M99448)
- **Footprint:** 515mm x 570mm (L22056, M66119), 550mm x 570mm (M99448)
- **Product Weight:** 14.4kg (L22056, M66119), 15kg (M99448).
- **Maximum User Weight:** 190kg (L22056, M99448), 160kg (M66119)

Installation Information:

Requires Assembly. Please refer to User instructions

Safety Precautions

Please ensure the user instructions are fully read and understood before installation/ use. Retain user instructions in a safe place for future reference.

PTDS029 Mobile Commode

Cleaning

Simply hose down/wash down or pressure wash with a mild detergent solution, ensuring all detergents are washed off thoroughly.

Wipe the commode thoroughly dry, paying particular attention to all joints and abutments. Do not use an abrasive cloth for drying

Caution: Do not use abrasive cleaners which could damage the powder coated surfaces or cleaners that could attack chromium plating as this will affect the integrity of the chromium plated surface.

It is recommended that all joints, abutments and castors are then sprayed with a water dispersing agent such as WD40; this will give your commode greater longevity and avoid premature rusting .

Care & Maintenance

Check all components while in service on a regular basis, once a week is recommended (and, following decontamination, prior to reissue).

Ensure that all components are secure.

This includes:

Nuts, screws and hand wheels. Tighten if necessary.

Condition and operation of castors including brake function.

Check all E-clips are securely engaged.

Check the condition and operation of footrests.

Check upholstery and moulding for splits.

Tighten and/or replace as necessary before re-use.

FAQ's

- **Q:** Can the commode be wheeled over a toilet?

A: The commode can be positioned over a standard height toilet.

- **Q:** How is the commode bucket removed for emptying?

A: The commode bucket slides out, on runners, at the rear of the commode.



- **Q:** Can the product be used in the shower?

A: No, the commode cannot be used in the shower

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK

User Instructions

L22056 M99448 N54411	NRS Wheeled Commode Fixed Height NRS Wheeled Commode Wide/Fixed Height NRS Economy Wheeled Commode		Max. User Weight
M66119	NRS Wheeled Commode Height Adjustable		

General Safety Warning

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

Intended Use

The Wheeled Commode range is designed to be used, either as a commode, or over the toilet. It may be used for short transfers on even floor surfaces to move the client over the toilet subject to a suitable risk assessment. It is suitable for less mobile clients. The products feature; removable arms to allow side transfers, detachable swing away footrests and four brake castors. Model M66119 is also height adjustable.

















Limitations of use

Do not use as an alternative to a wheelchair.
Do not use in the shower/ as a shower chair.
Do not use to transfer clients over raised thresholds.
The commode should only be used indoors.
If you have any doubts about the restrictions on use, please contact NRS (details on reverse).

Unpacking

Remove all items from the packaging, check all the items listed below are present (dependant on model type). Check each component for any visible damage. If you see any damage, or suspect a fault, do not assemble the product; but contact your supplier immediately

Contents:

	L22056 M99448 N54411	M66119	Qty.
Main Chassis (c/w Castors)			1
Main Chassis			1
Seat Cover			1
Detachable arms			2
Detachable back			1
Potty			1
Footrest			2
Adjustable Height Castors			4
Hand wheel			2
E - Clip			4

**Model:
M66119**



Assembly Instructions:

1 – M66119 Only

Fit the Adjustable height castors to the main chassis, slide the tube to the desired height, and check that all legs are equal in height. Secure with the E-clips provided, making sure each clip is fully connected.

Please note; the height should be set so the client feels comfortable, but ensure there is adequate clearance if being used over a toilet

2 – All models

Place the chassis on a level floor.

3 – At the rear of the commode there are two black threaded hand wheels. Undo the hand wheels so that the screw thread is clear from the inside of the vertical tubes. Place the tubes of the detachable back into the chassis tubes and secure by turning the hand wheels until you feel the resistance and the screw thread is biting against the leg of the backrest.

Caution: DO NOT over tighten the hand wheels. Do not use tools as it is only necessary for the back to be secured finger tight.

4 - Place the two detachable arms into the locating holes on either side of the seat moulding, ensuring they are fully inserted and secure.

Assembly continued:

5 - Position potty on supports brackets (if required) and place grey vinyl seat pad over the moulded seat.

6 - Clip the two footrests into position, by fitting them from the side, over the two studs on either side of the front of the commode frame and rotating them towards the front of the frame until they click into place. Check the “swing away” and “tilt up” operation of the footrests.

7 - Check the operation of the braked castors and that the commode runs smoothly on a level surface.

Caution: If you suspect a fault for any reason with your commode, do not use and contact your supplier immediately.

8 - The commode is now ready for use.

Safety precautions:

- Do not exceed Maximum User Weight.
- Always ensure the product is suitable for your requirements; if in doubt, check with a health professional.
- Always ensure all castors have the brake applied and are in a forward aspect (not trailing) and the footrests are up when sitting onto or rising from the commode to reduce the risk of tipping. Note: Footrests should only be used to support the feet and should not be used to stand on when entering or leaving the commode.
- Check that no parts of the body are outside the commode if passing through doorways.
- Take particular care when using the product on damp surfaces.
- Before every operation inspect the product to ensure it is fit for purpose; see Maintenance.
- Damaged, worn or missing components must be replaced before further use by an approved repairer.

Commode Bucket Cleaning

Note: protective gloves are recommended when emptying the commode potty.

To remove the commode bucket, simply slide the bucket out from the chrome runners at the rear of the commode. Cover the top of the bucket to avoid splashing. Carefully dispose of the contents into a proper sluice or toilet.

Caution: Never empty contents into an open drain, sink or basin. This could cause a serious health risk.

Clean out the residue with a suitable disinfectant and refit to the commode.

It is recommended that the seat is also wiped with a disinfectant wipe or mild solution after each use. Also wipe down the frame if any spillages are apparent using a mild disinfectant solution/wipe.

Note: NRS Healthcare offers a range of protective gloves and cleaning solutions to assist in the effective decontamination of the potty and commode.

For further information visit our website or contact customer services, details below.

Commode cleaning

Simply hose down/wash down or pressure wash with a mild detergent solution, ensuring all detergents are washed off thoroughly.

Wipe the commode thoroughly dry, paying particular attention to all joints and abutments. Do not use an abrasive cloth for drying

Caution: Do not use abrasive cleaners which could damage the powder coated surfaces or cleaners that could attack chromium plating as this will affect the integrity of the chromium plated surface.

It is recommended that all joints, abutments and castors are then sprayed with a water dispersing agent such as WD40; this will give your commode greater longevity and avoid premature rusting.

Maintenance

Check all components while in service on a regular basis, once a week is recommended (and, following decontamination, prior to reissue).

Ensure that all components are secure.

This includes:

Nuts, screws and hand wheels. Tighten if necessary. Condition and operation of castors including brake function.

Check all E-clips are securely engaged.

Check the condition and operation of footrests.

Check upholstery and moulding for splits.

Tighten and/or replace as necessary before re-use.

If in any doubt, please do not use or issue, but contact your supplier for service support.

Manufactured for

NRS Healthcare,

Sherwood House,

Cartwright Way,

Coalville,

LE67 1UB,

United Kingdom.

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.