

User Instructions: N85146 Home Helper Trolley – Flat Pack

General Safety Warning

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

1.0 Intended Use

The Home Helper Trolley is designed to enable users to safely transfer items and meals around the home; the lower tray is set forward on the frame to allow users to walk naturally when pushing.

2.0 Safety Precautions

WARNING this trolley is not designed to be sat upon.
WARNING although this item is used when walking
IT SHOULD NOT BE USED AS A WALKING AID.

3.0 Unpacking

Carefully unpack the product, check all parts are present, see pack contents below. Check for any damage. If you see, or suspect, a fault do not use. Contact your supplier.



Item	Pack Contents	No. Off
1	Frame Side Right	1
2	Frame side Left	1
3	Tray	2
4	Cross Tube Rear	2
5	Cross tube Front	2
6	E-Clip	4
7	Castor	4


4.0 Assembly Instructions

Note this product does not require tools for assembly
1- Place the frame side on the floor, as shown in diagram-1. A soft surface is preferred to prevent scratching.


2- Fit the cross tubes into the frame, note; there are four tubes, two rear tubes and two front tubes (with welded tabs), the front tubes must be fitted in the correct orientation, with the tabs facing forward and upwards see diagram 3.



Diag.2

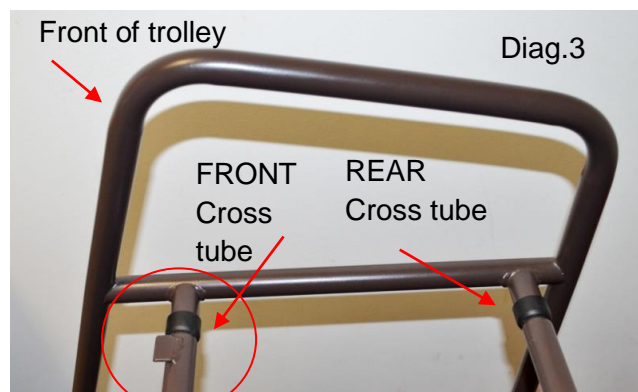

Enlarged View
showing Front Cross
tube with TABS
facing forward and
upward

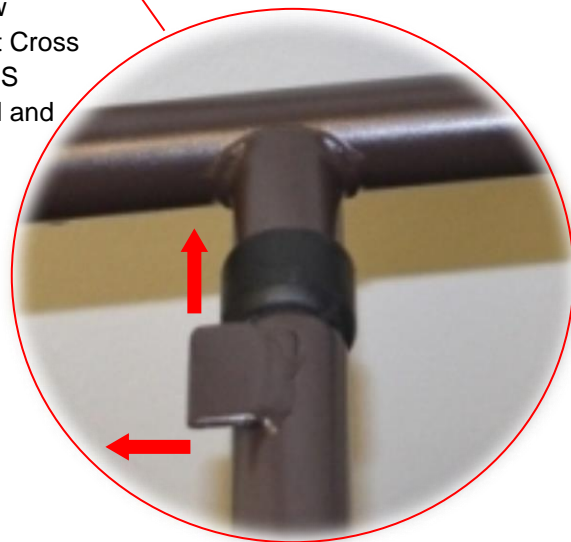


Assembly Instructions Continued

3- To engage in the frame, press in the brass spring button to allow the tube to enter the frame, push the tube until the spring button “clicks” into position and the button is visible through the hole in the frame. See diag. 4

4- Repeat for all cross tubes

Brass spring button engaged correctly in the frame side

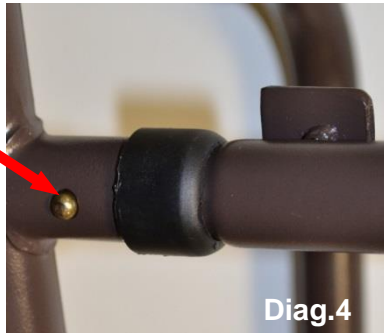

5- Align the other frame side onto the four side tubes and engage the spring buttons as described above, ensure all spring buttons have clicked into place and are visible through the holes in the frame.

6- Turn the trolley upside down and place on the handles. See diag. 5



7- Fit the trays: Note, fit the lower tray first.


8- With the tray sideways enter the frame and rotate into position see diag. 6



9- Engage the clips on the front of the tray onto the cross tube tabs. See diag. 7, and then clip the rear edge of the tray onto the rear tube. See diag. 8

See diag. 8

10- Repeat for the upper tray.

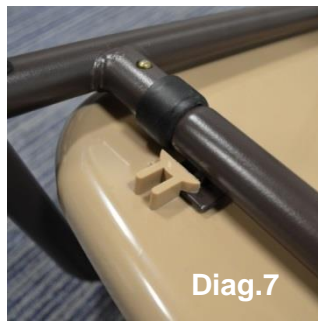

11- Fit the castors to the frame by pushing them into the bottom of the frame, align the holes in the frame with the holes in the castor leg and secure with e-clip. See diagram 9, repeat for all four castors, ensuring they are all at the same height. Turn product upright and check it is level and stable.

Note: Trolley height can be adjusted to suit the user, but ensure all castors are set to the same level and the trays are level

5.1 Cleaning Instructions

- Clean regularly, use a general purpose, neutral detergent and warm water
- Do not use abrasive cleaners or cloths as this will damage the surfaces of the product.
- Allow to dry thoroughly
- The trays are designed to clip on/off for cleaning. To remove, unclip the rear edge and lift clear. To refit, engage the clips on the front of the tray on the front cross tube and secure by clipping the rear edge of the tray onto the rear cross tube.
- Check tray is level and secure before use.

5.2 Maintenance

- Inspect the product regularly for signs of wear or damage, particularly the condition of the castors
- Check all fixing clips are inserted correctly and there is no excessive movement.
- If you see or suspect a fault do not use – contact your dealer immediately.



NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk