NOTICE
USER'S MANUAL
GEBRAUCHSANWEISUNG
MANUAL DEL USARIO
MUNKAMÓD
ÇALIŞMA MODU
MANUALE D'USO
UPUTE ZA UPORABU
MANUAL DE UTILIZAÇÃO
NAVODILA ZA UPORABO
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΤΗ
GEBRUIKERSHANDLEIDING
MANUALUL UTILIZATORULUI

CANNES ANGLAISES
ELBOW CRUTCHES
UNTERARMGEHSTÜTZE
BASTÓN INGLÉS
KÖNYÖKMANKÓ
KANADYEN
STAMPELLE
PODPAZUŠNA ŠTAKA
CANADIANAS
BERGLE
ΒΑΚΤΗΠΙΕΣ ΑΓΚΩΝΟΣ
ELLEBOOGKRUKKEN
CĂRJĂ CU SPRIJIN PE ANTEBRAT

Ref 920100
Evolution - Progress-Tiki

Ref 920201
Advance

Ref 920113
Tiki

A

Evolution
200 ***

Advance
20 *9**

Progress II
201 7**


Progress I
200 7**

Tiki Bleu-Rouge : 200 523
Tiki Anis-Noire : 200 522

Indicateur d'usure / Wear indicator / Verschleißanzeiger

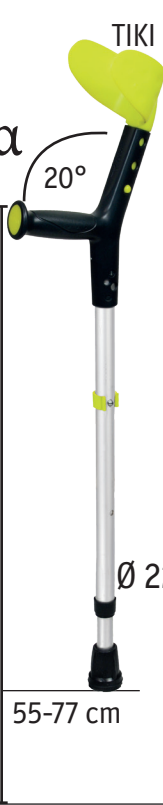

0,52 kg 140 kg

0,52 kg 130 kg

0,52 kg 150 kg

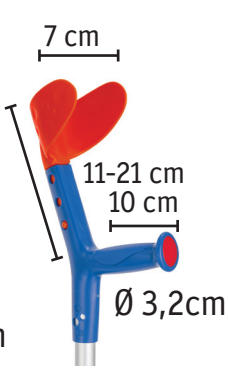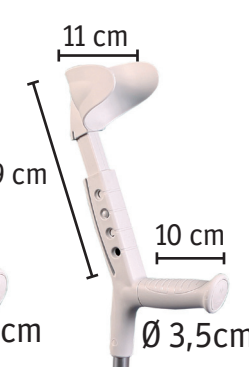0,52 kg 100 kg

1

2

3

4

77-98 cm

EVOLUTION
ADVANCE
PROGRESS I & II

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EN- Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below -15°C or 5°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips...).
- ♥ Check regularly if all parts are tightened (clips, handle, rubber tip..).
- ♥ Hold firmly and completely the handle when using.
- ♥ In case of falling : let the product fall down on the side, so that you don't fall on it.
- ♥ Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
- ♥ Don't use on wet or icy surfaces. (snow, ice, water..)!)
- ♥ Don't use in stairs, severe risk of injuries!!
- ♥ Use only as a walking aid and in proper way : don't use as a lever or to move furniture.
- ♥ ALWAYS observe the weight limit on the labelling of your walker.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.
- ♥ DO NOT hang anything on the Walking aid.
- ♥ In case you didn't use your walking aid for a long time, let it checked by the technical staff, before using.
- ♥ Be careful when using: beware holes and deformations in the floor.
- ♥ The suction cap must not be too worn down. If the cap shows considerable wear, ask your supplier or nursing staff to replace it (photo A).
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.
- ♥ The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. Intended use, indications, contraindications and installation

Intended use:

Walking aids are used for two purposes: as part of a rehabilitation programme when the user is recovering from an injury or operation and as a long-term aid to mobility when the user has a permanent difficulty with walking.

These devices help in maintaining balance, aid in the prevention of injuries (particularly falls), and allow the person to attend to his/her errands independently. Mobility aids can be used indoor or outdoor, depending on the model.

Mobility aids are used to modify the weight redistribution: some of the weight carried through the legs when walking is transferred through the arms of the frame or stick as it is leant on for support. Products equipped with a seat allow

short rest to the user.

Indications

- Loss of autonomy due to walking disorders - Balance and/or posture disorders
- Walking problems
- Rehabilitation programme after an injury or an operation

Contraindications Don't use in case of:

- Severe cognitive disorder - Severe loss of balance - Severe motor or perception disorders - Weakness in the upper limbs

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

Height Adjustment : Take out the clip from the tubes using both thumbs, then adjust the interior pipe to the desired height and lock again with the clip. Before using the crutch, check that the metal pin of the clip has gone through both pipes and exits from the outer pipe.

The crutches have 2 adjusting holes on the outside pipe and 5 holes on the inside pipe, what means 10 adjusting positions.

The crutch can be adjusted every 2.5 cm.

Keep the crutch close to your body while going to avoid accidents.

3. Cleaning

The crutch has to be cleaned regularly with a mild cleaning agent and a soft cloth. NEVER use OILY substances!! Slip hazard!!

Ask your retailer or the technical staff to check the walking aid regularly.

The rubber tips must be checked and changed regularly.

4. Materials

Tube in anodised aluminium and polypropylen handle. The Advance model has an handle covered with a soft material.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

This product has a 24 months warranty, from the date of purchase, against any manufacturing defect. Standards: DIN-EN-12182, EN-ISO 11334-1, DIN-EN-ISO 10993-1. Biocompatible. Duration of use: 2 years.

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