

# USER INSTRUCTIONS

## PADDED CORNER SHOWER STOOL

M18319

Padded Corner Shower Stool

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

### Maximum User Weight



### 1. INTENDED USE

The Padded Corner Shower Stool is height adjustable and is intended to be used in a shower to provide a stable, seated support, where a user is unable to stand for long periods (i.e. during showering).



### 2. KEY DIMENSIONS

Seat height Adjustable 510-660 mm (20-26")

### 3. SAFETY

**Warning:** The maximum user weight must not be more than 160 kg (25 st).

Do not exceed the maximum user weight limit.  
Inspect at regular intervals for wear and damage.  
Do not use if rubber feet are damaged.

### 4. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

### 5. HEIGHT ADJUSTMENT

1. Put seat upside down on floor/flat surface.
2. Adjust to the correct height by removing pin clips, sliding extension leg to required position and replace pin clip.

3. Ensure all three leg extensions are secured to the same adjustment hole to ensure even height.

### 6. USING THE SHOWER STOOL

Place in shower. Ensure all three legs are level and in contact with the shower floor. Sit centrally on the shower stool.

### 7. CLEANING INSTRUCTIONS

Clean regularly, use a general purpose, mild detergent and warm water.

### 8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal & plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

Household Waste and Recycling Centres.

### 9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)