

# User Instructions

## N72000 Contour Community Bed Rail

### General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

### 1.0 Intended Use

The Community Bed Rail provides a stable support to assist users getting into and out of bed. It can also be used to aid repositioning when lying in bed.

This product is designed for use with profiling metal framed community beds with 50mm X 25mm (2" X 1") rectangular side tubes.

Subject to a risk assessment this product can be fitted to the bed with most side rails already in place (excluding 'mesh side rails').

### 2.0 Safety Precautions

An individual risk assessment must be carried out for each user of the bed rail on a case by case basis.

The assessment must include the suitability of the bed rail with the user as well as the compatibility and position of the bed rail to the bed (including any fitted accessories such as side rails and cot bumpers).

**WARNING** - To avoid possible head entrapment, position the Bed Rail at least 320mm (12.5") away from both headboard and footboard, figure 2.

**DO NOT** exceed product Maximum User Weight 25st, 160kg

**ALWAYS** check the profiling operation of the bed with the bed rail already fixed in place for any potential risks prior to use.

Take care on beds with rising backrests. In some cases, when the backrest raises, the gap between the bed rail and bed could become less than 25mm.

To prevent hand entrapment do not hold the grab handle when altering the profile of the bed.

### 3.0 Unpacking

Remove packaging and check all parts carefully. If any parts appear damaged do not assemble and contact your supplier immediately.

Maximum User Weight



Figure 1 – N72000

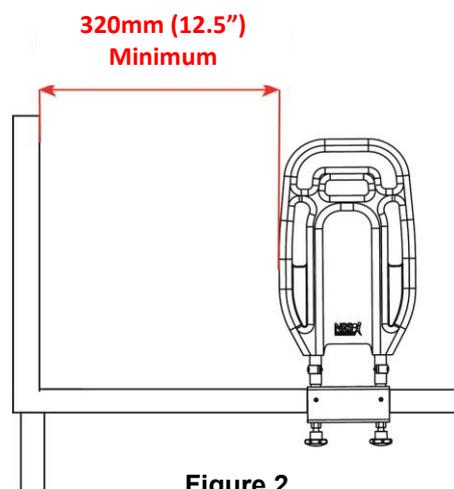


Figure 2

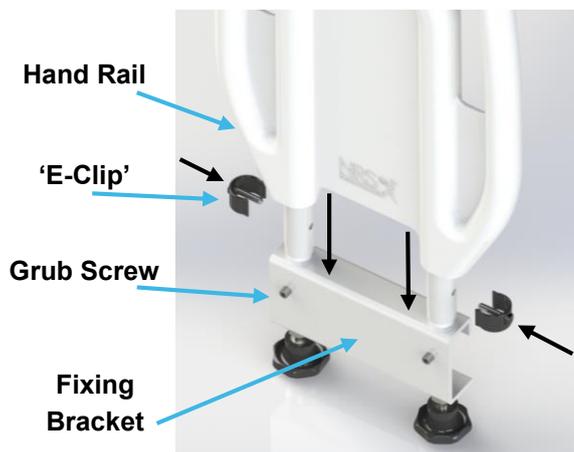


Figure 3

#### 4.0 Installation Instructions

Push the fixing bracket up into the tubes of the contour bedrail and secure using the 'e-clips' provided, figure 3.

Loosen both hand wheels to enable the fixing bracket to fit over the bed frame, figure 4.

Slide the bracket along the bed frame to the correct position for the user.

*This position should be based on a clinical assessment and should adhere to all safety precautions.*

To secure the fixing bracket to the bed frame, tighten both the hand wheels until they contact the frame and will not tighten any further.

Use the Allen key provided to tighten the grub screws on the front of the fixing bracket.

#### 5.0 Removal Instructions

**WARNING** - Do not detach the hand rail and leave the fixing bracket on the bed frame.

Remove the 'e-clips' and lift the hand rail off the fixing bracket.

Unscrew the grub screws using the Allen key provided.

Loosen the hand wheels to disengage the fixing bracket from the bed frame.

Lift the fixing bracket off the bed frame.

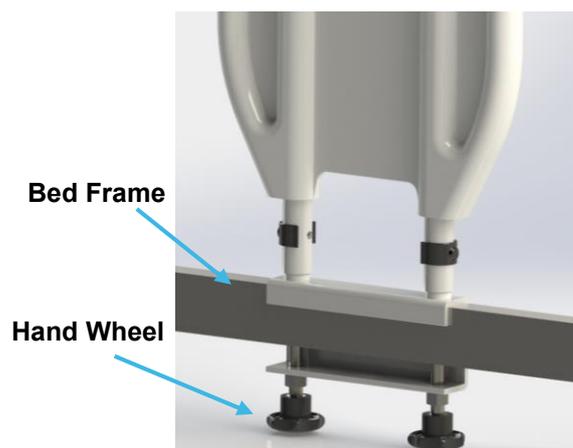


Figure 4

#### NRS Healthcare

Sherwood House, Cartwright Way,  
Forest Business park, Bardon Hill,  
Coalville, Leicestershire, LE67 1UB, UK

#### 6.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow product to dry fully prior to use/storage.

#### 7.0 Maintenance

Check the 'e-clips' are inserted correctly and that there is no excessive movement.

Check the hand wheels and grub screws are tight and the bed rail is securely fitted to the bed frame.

Inspect the product regularly for signs of wear or damage, or if the bed rail is loose.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

#### 8.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

#### 9.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: [customerservices@nrshealthcare.co.uk](mailto:customerservices@nrshealthcare.co.uk)

**NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:**

**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**