

USER INSTRUCTIONS

2-BAR SIDE RAIL BUMPER

S58322

2 Bar Side Rail Bumper

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

To fit the NRS Community Bed 2 Bar Steel Side Rails S39200. They provide additional comfort and security to the user.



2. SAFETY

Warning: The Bumpers are only designed for use with NRS Community Beds (models: P78269, P78271, P78283, P78295, when fitted with 2 Bar Side Rails S39200). Do not use on any other make or model of bed.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

3.1 Contents:

1. Cot Bumper x 2

4. INSTALLATION



1. Drape the bumper over the bed rail as shown.



Make sure that the Velcro tabs are on the outside of the bed and the product label is uppermost.



2. Fasten the top part of the bumper using the Velcro strips as shown.



3. Fasten the lower part of the bumper using the Velcro strips as shown.
4. Repeat steps 1 to 3 for the opposite side of the bed.



Check that the bumper is fully secure before use.

5. OPERATION



The cot bumpers have been designed to be raised and lowered with the side rails, they do not need to be removed.

Check that the side rails operate correctly in accordance with the side rail user instructions.

The side rails should move freely up and down fully without obstruction.

6. MAINTENANCE

Regularly check the cot bumper for damage.

Regularly check the mesh and Velcro strips for fraying or damage.

7. CLEANING INSTRUCTIONS

Clean regularly using a general purpose, neutral detergent and warm water applied to a damp cloth.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

Allow product to fully dry prior to use/storage.

8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from polyester fabric and polyurethane foam and can be disposed of

through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk