

# USER INSTRUCTIONS

## MOWBRAY TOILET SEAT & FRAME RANGE

### EXTRA WIDE

M48805	Mowbray Frame & Seat Extra Wide
M48817	Mowbray Frame & Seat Extra Wide – Floor Fixed

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

### Maximum User Weight



## 1. INTENDED USE

The Mowbray toilet frame and seat is designed to assist those who have difficulty in lowering onto and rising from a standard toilet. The plastic moulded seat offers support in a natural position and the frame is height adjustable.



## 2. SPECIFICATION

1. One piece moulded plastic seat.
2. White epoxy coated frame.
3. Non-slip ferrules (free standing versions)

## 3. SAFETY

**WARNING:** The maximum user weight must not be more than 254Kg (40st).

**DO NOT:** attempt to stand or climb on the product.

**MAKE SURE:** all legs are set at equal height to avoid rocking.

## 4. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.


## 5. INSTALLATION

**Note:** The free standing versions of this product are delivered ready assembled. If your product is not assembled, please follow instructions.

Fit the moulded toilet seat to the frame by clipping onto the front and back cross tubes as shown in the picture above.

### To adjust the height:

1. Remove the 'e-clips', slide the legs to the desired height, align the holes and re-insert the 'e-clips'.
2. Position frame around toilet.
3. Model M48817 only, secure the frame to the floor by screwing down the fixing legs.

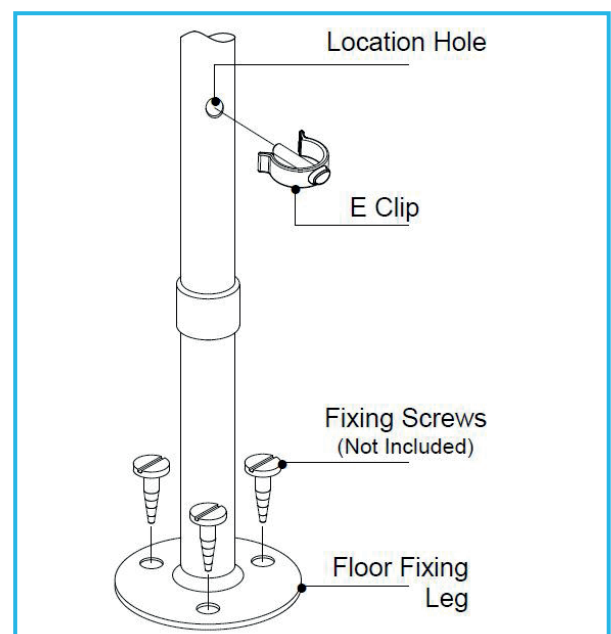

**Note:** It is important to set all four legs to the same height to prevent the frame from rocking. Height adjustment is not possible once the legs are secured to the floor

## 6. MAINTENANCE

- Check that all 'e-clips' are inserted correctly and floor fixing screws are secure. There should be no excessive movement.
- Regularly check the condition of the ferrules.
- Periodically inspect the product for signs of general wear or damage. Report any suspected faults to supplier and remove the product from service.

## 7. CLEANING INSTRUCTIONS.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

Regularly clean with a non-abrasive cleaner.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

This product is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 70°C.

Allow product to fully dry prior to use/storage.

## 8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal & plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

Household Waste and Recycling Centres.

## 9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)