

USER INSTRUCTIONS

SHELFORD SUPPORT FRAME

G21789

Right-Hand Side Frame

G21791

Left-Hand Side Frame

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

This multi purpose support frame can be used individually, as a bath rail or toilet support, or in pairs on either side of the toilet to aid the user to sit or stand with greater ease and stability.



2. SAFETY

Warning: The maximum user weight must not be more than 190 kg (30 st).

Check the product for signs of damage prior to fitting, report any defects to your supplier or NRS immediately.

It is recommended that this product should only be installed by a suitably trained and qualified person.

Screws and fixings are not supplied with this product, it is recommended a survey of the floor/ flooring materials is carried out to ensure the correct type of fixings are used.

Note: It is recommended in a wet environment that stainless steel screws are used to avoid corrosion.

For hygiene and safety purposes check the product regularly for any signs of wear or damage, paying particular attention to the surface coating.

Check regularly that all screws and fixings are secure and the frame is stable.

Note: This product should only be used for it's intended purpose.

Maximum User Weight



3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. FITTING INSTRUCTIONS

- Place the frame/frames in the desired position
- Ensure the floor is of suitable construction
- Mark through the holes in the frame
- Drill and plug holes with suitable fixings
- Using suitable screws fix the frame, making sure all screws are firmly fixed in position
- Before use check the frame is firm and stable.

5. CLEANING INSTRUCTIONS

The frame & seat may be cleaned using a non-abrasive cream cleaner or detergent solution.

6. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

7. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk