

Product Technical Data Sheet 047

P01560 Height Adjustable Shower Chair

Product Range

N/A | N/A

Product Description & Intended Purpose

This adjustable height shower chair is intended to be used in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering.

Product Features

- A tubular steel frame with a hardwearing, rust-resisting coating and a moulded polypropylene (PP) seat.
- Arm and back rests provide additional support and comfort.
- Integrated drainage holes increase safety and comfort.
- Rubber ferrules and splayed legs to increase stability on wet surfaces.
- Height adjustable.



Technical Specification

Dimensions (mm)

- | | |
|----------------------------------|---|
| • <u>Seat Height min:</u> 450 | • <u>Arm Rest Height from Seat:</u> 200 |
| • <u>Seat Height max:</u> 600 | • <u>Backrest Height:</u> 370 |
| • <u>Seat Width:</u> 440 | • <u>Footprint Max:</u> 620 X 500 |
| • <u>Seat Depth:</u> 400 | • <u>Footprint Min:</u> 620 X 450 |
| • <u>Width between arms:</u> 500 | <u>Maximum User Weight:</u> 160kg/25st |

Installation Information: *(Note: for detailed information refer to User Instructions)*

- Fit the seat to the frame by first clipping it onto the rear of the two closely spaced cross bars, and then onto the front cross bar.
- Fit the legs onto the frame and adjust by sliding to the desired height and secure with the fixing clips.
- Set all legs to the same height.
- Check the seat is level and the product is stable before use.

PTDS047

P01560 Height Adjustable Shower Chair

Safety

- It is recommended that before use an inspection of the product and location suitability is carried out by a suitably qualified person.
- Do not exceed the maximum user weight 160Kg (25st).
- Ensure seat and legs are correctly fitted and secure prior to sitting on the chair.
- Avoid extending the legs in a manner which causes the stool to slant.
- Do not move the stool whilst occupied.

Cleaning

- Clean regularly, use a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.
- This shower chair is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 80°C
- Dry the seat thoroughly before use.

Care & Maintenance

- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the feet/ferrules.
- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Check for general wear or damage.

FAQ's

- How do I decide what height to set the seat at?

To measure the height required, sit on a chair with your knees and hips at right angles with your feet flat on the floor (e.g. a 'dining chair'). Measure from the floor to behind the knee. The shower chair can then be set to this height or as close as possible.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK

**Maximum
User
Weight:**



General Safety Warning

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

1.0 Intended Use

This shower chair is intended to be used in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand for during showering.

2.0 Safety Precautions

- Do not exceed the maximum user weight.
- Ensure seat is correctly fitted and secure.
- Inspect product at regular intervals, see section 5.2
- Do not use if feet are damaged.
- Ensure all legs are secured at the same height to keep the chair stable.
- Do not attempt to move the chair when occupied.

3.0 Unpacking

Check all parts carefully before assembling the product. Do not assemble if any parts are damaged.

4.0 Assembly and Height Adjustment

- Where the seat is supplied separately, fit to the frame by first clipping it onto the rear of the two closely spaced cross bars, and then clipping to the front cross bar.
- Adjust the legs by sliding to the desired height and secure with the fixing e-clips.
- Set all legs to the same height.
- Check the seat is level and the product is stable before use.



5.0 Product Care

5.1 Cleaning Instructions

- Clean regularly, use a general purpose, neutral detergent and warm water
- Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.
- These seats may be autoclaved on cleaning cycles up to 80°C.
- Dry the seat thoroughly before use.

Product Care - Continued

5.2 Maintenance

- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.
- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Check for general wear or damage.
- If you see or suspect a fault do not use and contact your dealer immediately.

6.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

From the NRS Healthcare Range, you may also like:

Nuvo Adjustable Bath Step
M11077



Spa Curved Grab Rail
M53527 Length: 355mm (14")
M53539 Length: 480mm (19")
M85553 Length: 620mm (24 1/2")



Combi Grabber
N72358 Length: 660mm (26")
N72369 Length: 813mm (32")



NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business Park, Bardon Hill,
Coalville, Leicestershire.
LE67 1UB UK

Tel. Main Office: 0330 100 00 98
Tel. Customer Services: 0345 121 8111
Email: customerservice@nrshealthcare.co.uk



NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk