

User Instructions

M99631 NRS Foldaway Footrest

General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

Intended Use

This height adjustable footrest offers a choice of three different height positions. The highest position elevates the legs to help stimulate circulation and ease tired legs. The footrest can be angled in various positions to rest legs comfortably. The footrest can also be used flattened to a neat footstool for resting thighs and keeping feet warm.

Unpacking

This product is supplied fully assembled. Remove from packaging; Check for damage. If you see any, or suspect a fault, do not use the product; contact your supplier immediately. Please Note: The front bar is tucked inside the fabric cover to prevent damage in transportation. Before first use remove the bar from inside the cover and allow to swing freely.

Safety Precautions

- Do not use this product until you have read these instructions
- For maximum stability ensure the hinge on the base is facing the chair as shown in Diag.1
- DO NOT SIT on the footrest, The product has a maximum weight limit of 18kg (2.8st) and is not suitable for sitting on

User Instructions

1 - Position the footrest as shown in Diagram 1. The footrest can then be set at the required height and angle. Note the angle can be either forward or backward facing, whichever is the most comfortable.


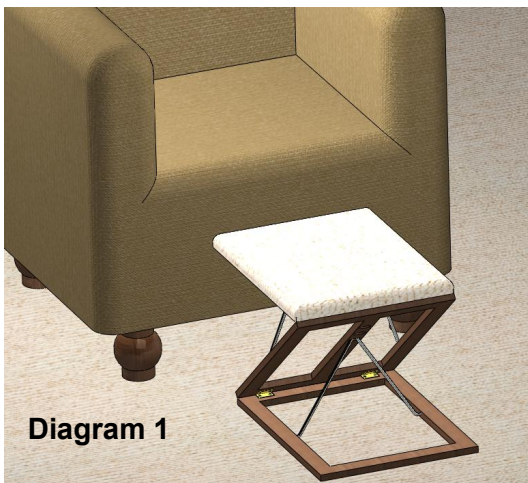


Diagram 1

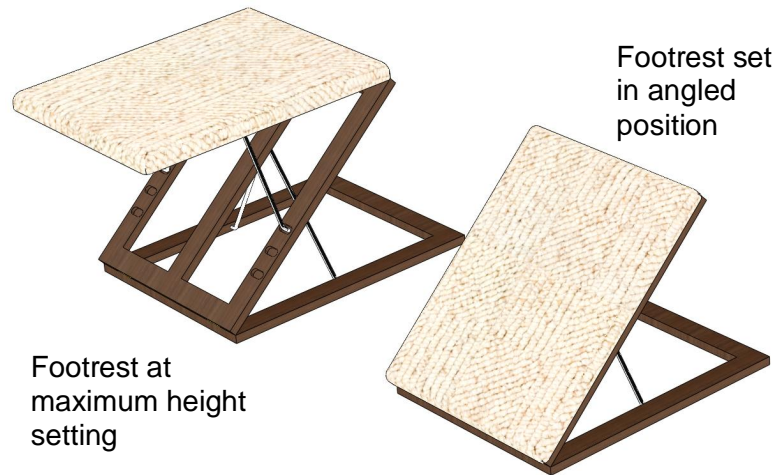

Footrest at maximum height setting

Footrest set in angled position

2 - When the footrest is set in the desired position, the front and rear bars must be engaged in the appropriate slots as shown in diagram 2 & 3

Diagram 2


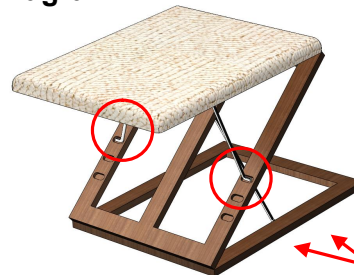


Diagram 3

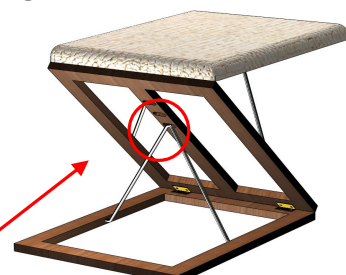

Engage bars in slots as shown - (3 places)

Product Care: Cleaning

The frame assembly should be wiped down with a cloth, dampened with a mild detergent solution. The top cover can be removed for cleaning - HAND WASH ONLY, using a mild detergent.

Product Care: Maintenance

Periodically check that the support bars and hinges are in good working order

The **Foldaway Footrest** is part of a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range please visit our website www.nrs-uk.co.uk

or request a catalogue from the number below.

NRS Healthcare

Sherwood House, Cartwright Way
Forest Business Park, Bardon Hill
Coalville, Leicestershire. LE67 1UB UK
Tel: 0845 120 4522 Fax: 0845 121 8112
Email: customerservice@nrs-uk.co.uk

www.nrs-uk.co.uk