

User Instructions

M11209 Folding Commode

General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

To assist those who have difficulty in accessing or using a standard toilet and is particularly useful for travel. It has a sturdy steel frame, moulded seat, arms and backrest, providing both comfort and security for the user. It folds easily for storage and transport, and is supplied with a five-litre potty and lid.

2.0 Safety Precautions

- Check the product has been unfolded correctly and both plastic catches are locked in position before use.
- Always place the commode on a sturdy, level surface.
- Check the product is stable before every use.
- Do not exceed the maximum user weight.

Maximum
User
Weight



Figure 1 – Product Unfolded

3.0 Unpacking

Remove packaging and check product carefully before assembling. Do not assemble if any parts are damaged, contact your supplier.

4.0 Assembly & Use

- To unfold the commode, simply hold the backrest and pull the seat downwards until it is fully open and the seat is horizontal.
- When fully open secure by engaging the two plastic catches on either side of the frame and lock in position, see figure 3.
- To fold the commode, simply reverse the procedure.
- To fit the potty slide in position under the seat, from the rear, making sure it is located on both rails underneath the seat.



Catch Unlocked



Catch Locked



Figure 2 Product Folded

Figure 3

Assembly & Use continued

- To remove the potty, slide out and fit the lid to avoid splashing.
- Carefully carry the potty and dispose of contents into a proper sluice or WC.
- **Never use an open drain, sink or basin. This could cause a serious health risk.**
- Clean out any residue with a suitable disinfectant and refit to the commode after use.

5.0 Cleaning - General

- Clean regularly using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage surfaces and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.
- Allow the product to dry thoroughly prior to use/storage.

6.0 Maintenance

- Check both seat catches are engaged correctly and there is no excessive movement.
- Inspect product regularly for signs of wear or damage.
- Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

7.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business Park, Bardonia Hill,
Coalville, Leicestershire, LE67 1UB, UK

Tel. Customer Services: +44 (0)345 121 8111
Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.

**Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk**

UL/M11209/08/23/Iss4