

USER INSTRUCTIONS

HEIGHT ADJUSTABLE SHOWER CHAIR

M48295

Height Adjustable Shower Chair

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

For use in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering.



2. SAFETY

WARNING: The maximum user weight must not be more than 160 kg (25 st).


3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

3.1 Contents:

1. Seat x 1
2. Frame x 1
3. Leg Extension x 4
4. Stainless Screw x 4
5. Nylon Washer x 8

4. ASSEMBLY/INSTALLATION

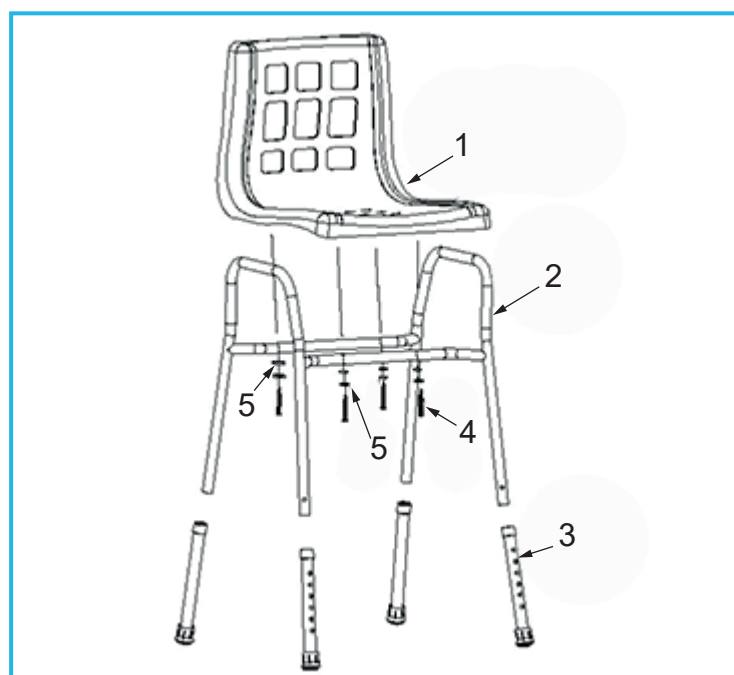

1. Fit the seat to the frame as shown using the four stainless screws, each fitted through two nylon washers and inserted into the seat moulding up through the frame. Do not over-tighten, hand tight is sufficient.

2. Fit an extension leg onto each frame leg. Adjust the legs to give the required height for the user (this should be such that the user's feet sit squarely on the floor with the thighs supported). Ensure the brass buttons protrude fully through the adjustment holes in the extension legs.

3. Place chair in required position and check it feels stable and secure before use.

5. MAINTENANCE

Periodically check the stability and rigidity of the product and, where applicable, that screws are tightened and in good condition. If you see any damage or suspect a fault, please contact your supplier.

6. CLEANING INSTRUCTIONS

- Clean regularly, wiping down using a soft cloth and a mild detergent/disinfectant solution.
- Do not use abrasive cleaners.
- Do not autoclave.
- Allow product to fully dry prior to use/storage.

7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk