# Spa Range



# FPN Adult Slipper Pan with Handle and Lid

Long Life Re-usable Polypropylene Adult Slipper Pan for use in Nursing, Rehabiltation, Elderly Care



The Spa Range is a range of re-usable living aids designed for use in Hospital Wards, Nursing and Care Homes, Rehabilitation and Personal Use at home.

Our Spa Range is an attractive, stylish and functional group of products which includes drinking cups, beakers, jugs, vomit bowls, medicine cups, urinals, bedpans and commodes.



Warwick House, Heathcote Way, Warwick, CV34 6TE

T: +00 44 (0)1926 422427 F: +00 44 (0) 1926 315405

E: info@sasco.co.uk W: www.sasco.co.uk

#### **Product Benefits**

- Suitable for both male and female use.
- Ideal for patients confined to bed.
- Lightweight, strong and durable.
- Secure fitting lid to prevent leaks and spills and reduces cross infection and unpleasant odours.
- Can be cleaned in automatic washer disinfectors or hand washed.
- Smooth material surfaces ensure ease of use and ease of cleaning.
- Suitable for use with a body weight up to 160kg.

# **Product Specification**

- Slipper Bedpan with Handle and Lid 1500ml Capacity
- 449mm x 309mm x 99mm
- Pack Size 12
- Registered as a Class 1 Medical Device (CE Marked)
- Complies with following Standards:
  - ISO 9001:2008

Quality Management System

- ISO 13485:2003

Medical Devices - Quality Management System

- BS 2588:1992

Reusable Portable Sanitary Pans

Manufactured from high quality Polypropylene

## **Guide to Using Spa Products from Warwick SASCo**

#### Usage

- For both male and female use.
- For use in a supine or seated position.
- Close lid after use to reduce odour and risk of spills.
- Empty after each use.

#### Cleaning

- Product should be washed before first use. Wash in warm soapy water and allow to dry naturally.
- Can be cleaned in automatic washer disinfectors compliant with requirements of ISO15883. Suitable for hand washing using detergents and disinfectants.
- Spa Products are designed for a repeated use guaranteed for 1000 washes.
- Scouring and abrasive cleaners will reduce the performance of products.

## Storage

- This product has a shelf life of 36 months from date of manufacture providing the recommended storage conditions are adhered to.
- Product should be stored in its primary packaging, off the floor on a pallet or shelf, and in a clean dry ventilated room at a temperature of 4 °C - 25 °C.