

### Product Range

N/A | N/A

### Product Description & Intended Purpose

To assist those who have difficulty lowering onto or rising from a standard toilet. Height and width adjustable to suit a variety of users. A chute is incorporated into seat design to simplify disposal and improve hygiene.

### Product Features

- Padded armrests provide extra comfort.
- Smooth and contoured plastic seat provides extra comfort and thigh support.
- Arms extend forwards to further aid the rising and lowering motion.
- Seat lid improves hygiene and aesthetics.
- Non slip rubber ferrules provide stability.



### Technical Specification

#### Dimensions (mm)

- Seat Height: 450 — 560 (17¾" - 22")
- Seat Width X Depth: 406 X 450 (16" X 17¾")
- Seat Aperture: 260 X 220 (10" X 8½")
- Frame Height: 680 — 790 (27" - 31")
- Width Between Arms: 490 — 615 (19" - 24")
- Footprint Maximum: 675 X 585 (26½" X 23")
- Footprint Minimum: 550 X 543 (21½" X 21")
- Maximum User Weight: 190Kg (30st)
- Product Weight: 6.3kg (14lb)

### Installation Information: *(Note: for detailed information refer to User Instructions)*

- This product is delivered flat pack and requires assembly before use.
- Assembly time is approximately 5 minutes.
- No specialist tools or skills are required.
- It can be adjusted for height and width to suit the individual user.

# PTDS001

## N53909 Mowbray Lite Plus

### Safety Precautions

- Do not exceed the maximum user weight of 30st 190kg.
- Do not attempt to stand or climb on the product.
- Ensure all legs are set at equal height to avoid rocking.

### Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.
- Always allow the product to dry fully prior to use/storage.

### Care & Maintenance

- Check that all fixing clips are inserted correctly and that there is no excessive movement.
- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.
- Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

### Accessories / Related Products

- F11572 Floor Fixing Kit
- F17938 Spare Ferrule

### FAQ's

- Can you sit on the product with the lid down?  
*Yes, the seat and lid have been tested to withstand the maximum user weight 30st (190kg).*
- Can it be used on all toilets?  
*Yes, the seat is height and width adjustable and it has a shorter chute which makes it more suitable for modern toilets.*

### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire, LE67 1UB UK