
## Intended Use

- To help overcome a disability by providing a platform to transfer into the bath, enabling the user to bathe or shower independently.

## Assembly Instructions (see illustration 1)

1. Lay the bench seat, bottom up, onto a flat surface, preferably a table.
2. Insert the armrest tube (A) into the armrest bracket (C). Ensure the buttons of the spring pin locate fully into position and protrude through the holes. The armrest should curve away from the bench when assembled correctly.
3. The backrest may be positioned on either side of the bench, allowing the bench to be used in either direction, dependent on the location of the bath tub. Decide on the correct position for the backrest, in relation to the bath tub, and insert the backrest support tubing (B) into the brackets (D) of the seat frame. Ensure that the backrest tubing is securely extended into the front and rear brackets and the spring pin buttons locate fully into position and protrude through the holes.
4. Place extension legs (E) into leg frames (F), ensuring that the suction feet are located into the legs underneath the handle.
5. Position the bench into the bath tub with the armrest and suction feet on the inside of the tub. Adjust the height of the legs, removing the bench from the bath tub if necessary, so that the seat is level and is not resting on the edge of the bath. Note, the legs inside the bath tub can be adjusted so that they are shorter than the legs on the outside of the bath tub. Ensure the buttons of the spring pins for each leg locate fully into position and protrude through the holes.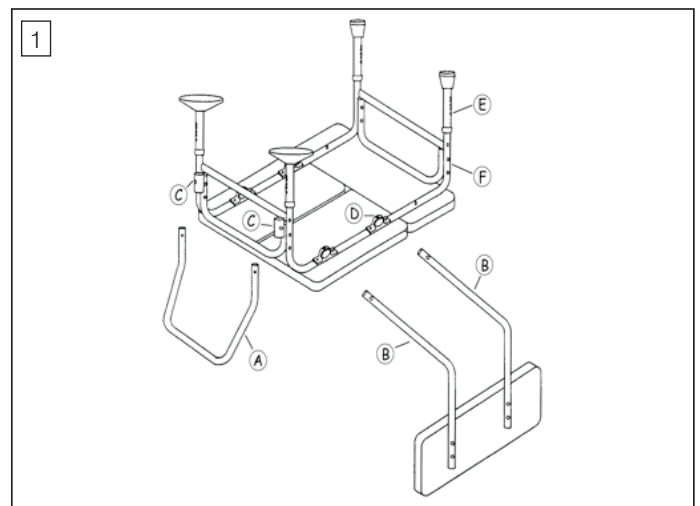

## ! Cautions

- Ensure the transfer bench is securely assembled and the feet and suction feet are in full contact with the floor and bath tub respectively, before use.
- Use of a slip-resistant bath mat in conjunction with the bench is strongly recommended.
- Regularly inspect all parts carefully for damage or wear, checking the security of any adjustable fittings.
- The Homecraft Transfer Bath Bench is not to be used to support more than 120kg / 264lb / 18¾st.
- Under no circumstances should you stand on the transfer bench.

## Care and Cleaning

- All parts can be wiped clean using a solution of mild detergent or household disinfectant.
- Dry thoroughly after each use.

Note: The ferrules on this bench are made from natural rubber and contain latex

Maximum user weight

Serious events experienced from using this product should be reported immediately to Performance Health and to the local Competent Authority.

Symbols Glossary located at <http://www.performancehealth.com/medical-device-glossary>.

Please retain these instructions for future reference.

**Performance Health International Ltd.**  
Nunn Brook Road  
Huthwaite, Sutton-in-Ashfield  
Nottinghamshire, NG17 2HU, UK  
Tel: +44 1623 448 706  
[www.performancehealth.co.uk](http://www.performancehealth.co.uk)

Performance Health France  
13 rue André Pingat CS 10045  
51 724 Reims Cedex France

### Distributed in the United States By:

Performance Health Supply, LLC  
W68 N158 Evergreen Blvd  
Cedarburg, WI 53012, USA  
+1800-323-5547  
[www.performancehealth.com](http://www.performancehealth.com)

### Distributed in Australia By:

Ausmedic Australia, Trading as  
Performance Health ANZ  
Unit 3, 3 Basalt Road  
Pemulwuy NSW, 2145, Australia  
1300 473 422  
[www.performhealth.com.au](http://www.performhealth.com.au)

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