

USER INSTRUCTIONS

STEP STOOL AND HANDRAIL

F19121

Step Stool

L64087

Step Stool with Handle

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

This product is available as a simple step stool or with a fixed handrail to offer the user additional support. It is ideal for general household use and especially suitable for helping users enter and exit the bath, or for use in the kitchen.

2. SPECIFICATION

Sturdy coated tubular steel frame fitted with non-slip rubber ferrules and a fluted rubber, slip resistant, covered wooden top.



6. CLEANING INSTRUCTIONS

The Step Top can be cleaned with soap and water and the frame with a non-abrasive cream cleaner or detergent solution.

7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal, wood and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

3. SAFETY

Warning: The maximum user weight must not be more than 140 kg (22 st).

Ensure an even floor surface to avoid rocking.

Check all fixings are secure, before and after use.

Check the product monthly for signs of wear and tear. Pay particular attention to the rubber feet.

4. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

5. MAINTENANCE

- Inspect the step at regular intervals, ensure the product is stable and rigid.
- Check all fixings are fully tightened and in good condition.
- Check ferrules monthly for signs of wear.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk