

warranty details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.




Drive Medical Ltd

Ainley's Industrial Estate, Elland,
West Yorkshire, GB HX5 9JP
www.drivemedical.co.uk

swivel cushion



www.drivemedical.co.uk

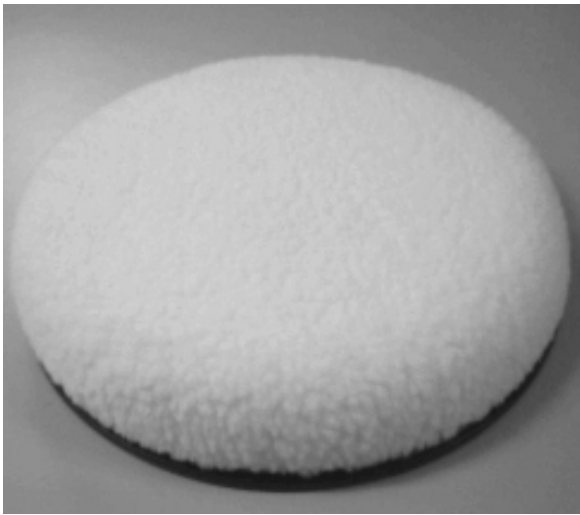

specification

The Drive Medical Swivel Cushion is designed to aid people who have mobility problems to turn on a number of different types of seat. It comes with a removable polyester fleece cover.

The Drive Medical Swivel Cushions are suitable for users up to 19 stone (120kg) and have a generous 40cm (16") diameter

They are suitable for many applications including dining chairs and car seats.

safety precautions

Apply the following precautions before use:

- Before using the cushion, ensure that it is placed on a surface which enables safe and secure seating for the user.
- Do not use in any environment where there is a risk of the cushion or user slipping from the seat (for example, where the seat is on a steep angle or if the cushion slides about on the seat).
- Carelessness Causes Fire. Always keep away from ignition sources such as cigarettes and portable fires.
- If used as a transfer device, we recommend you consult your therapist or health professional for guidelines about using the product.

care and maintenance

- The cover on the swivel cushion is removable and can be machine washed at 30°C (86°F).
- Periodically check that the cushion swivels easily and there are no cracks, etc in the base.