

Grip-On Bed/Chair Raiser K100, K200 & K300



1. Grip-On Bed/Chair Raisers are supplied fully assembled.

Minimum & maximum leg sizes accepted -

K100 - 19 to 38 mm

K200 - 32 to 50 mm

K300 - 43 to 65 mm

2. Grip-On raisers will fit castors and legs of many different shapes, the Grip-On must support its own weight to be working correctly. If the Grip-On falls straight off it is too large for the leg or castor.
3. Press down on bed or chair being raised to lock Grip-On in place.
4. If required set height adjustment by removing one or both base sections from each raiser. (height must be equal for all legs) Before use, test for stability.
5. Grip-On Raisers are designed for use in static mode only. As with all devices designed to assist the elderly or disabled, care should be taken to avoid sudden or undue movement, when using this equipment.

Maximum operating load

 500-kg per set of four raisers (including weight of chair/bed)

 **Langham**
Products



TRENT LANE
CASTLE DONINGTON
DERBY DE74 2NP
ENGLAND
TEL: (01332) 850277
INT: + 44 1332 850277
FAX: (01332) 850366
INT: + 44 1332 850366

K100/0401