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Directions for use

Please read the instructions before you start using this product

Intended use

For optimal safety the shower stool can be provided with a backrest.

The backrest is made of polyurethane, a material with good non-slip properties.

Assembly – see drawing

Detach the four snaplocks from the backrest. Mount the backrest by inserting the tubing through the rings on the underside of the shower stool. Reattach the snaplocks, one on each side of the rings. Be sure to insert them correctly into the tubes.

Maintenance

To be wiped clean after use. Use domestic cleaning materials without abrasive or corrosive properties. Solvents and chlorine should not be applied.

The backrest slats can be detached for cleaning. Polyurethane can tolerate temperatures up to 70°C.

The backrest frame is made of stainless steel material.

Warning

- Max. load on stool with backrest: 150 kg.
- Only use the backrest as described in the paragraph "Purpose".
- When the backrest is mounted in position it is not necessary to place the stool against a wall.
- Check the correct position of the snaplocks before use.
- Check the condition of the snaplocks frequently. Regularly check that the snaplocks are correctly inserted into the tubing.

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