



**THE LOCOMOTOR RANGE**



## **ROTA CUSHION INSTRUCTION SHEET**

### **LOCOMOTOR ROTA CUSHION**

**LOCO-068**      **Standard**  
**LOCO-078**      **Large**

### **DESCRIPTION**

The Rota Cushion is a soft padded, flexible turning disc which allows a smooth seated swivel transfer. It is ideal for use on car seats, chairs and beds.

### **INSTRUCTIONS FOR USE**

Place the Rota Cushion on car seat, chair, bed or similar. Sit centrally on to the Rota Cushion. Utilising the Rota Cushion rotate the whole body to the desired location. The swivel action of the Rota Cushion allows you to rotate smoothly into the correct position on the seated surface with minimum effort.

### **IMPORTANT**

**FOR SEATED TRANSFERS ONLY – DO NOT USE TO STAND ON.**

**ALL PATIENT HANDLING OPERATIONS SHOULD BE CARRIED OUT IN ACCORDANCE WITH APPROVED MANUAL HANDLING TECHNIQUES AND AFTER A FULL ASSESSMENT OF THE PATIENT AND THE ENVIRONMENT IN WHICH THE OPERATION IS TO TAKE PLACE.**

### **DIMENSIONS:**

Standard – 380mm diameter  
Large – 450mm diameter

### **MATERIALS:**

Cotton outer. Nylon inner.  
Steel rotating disc with hook and loop attachment.

### **CARE AND USE**

Check device integrity before use – do not use if soiled or damaged.

**Remove steel rotating disc before washing. Keep steel disc safe and replace when outer layers have been washed and dried. Do not submerge steel disc in water. If necessary wipe steel disc clean with alcohol and dry thoroughly.**

Outer fabric layers of Rota Cushion may be washed at 71°C and cool tumble dried.

For more information please contact the company you purchased the product from or contact Select customer services at:



Select Healthcare (U.K) Ltd.  
Unit 2, Albert Rd  
Rushden  
Northants  
NN10 0BU

Tel: (01933) 411729  
Fax: (01933) 410213  
email: [info@selecthealthcare.co.uk](mailto:info@selecthealthcare.co.uk)  
website: [www.selecthealthcare.co.uk](http://www.selecthealthcare.co.uk)