




4000 Series Seats Installation instructions

Colour: White to RAL 9016 / Grey to RAL 7035 or High Contrast Blue to RAL 5005

Internal Building use only

Safe Handling During Installation

To avoid the risk of damage to the seat during installation follow these wise precautions

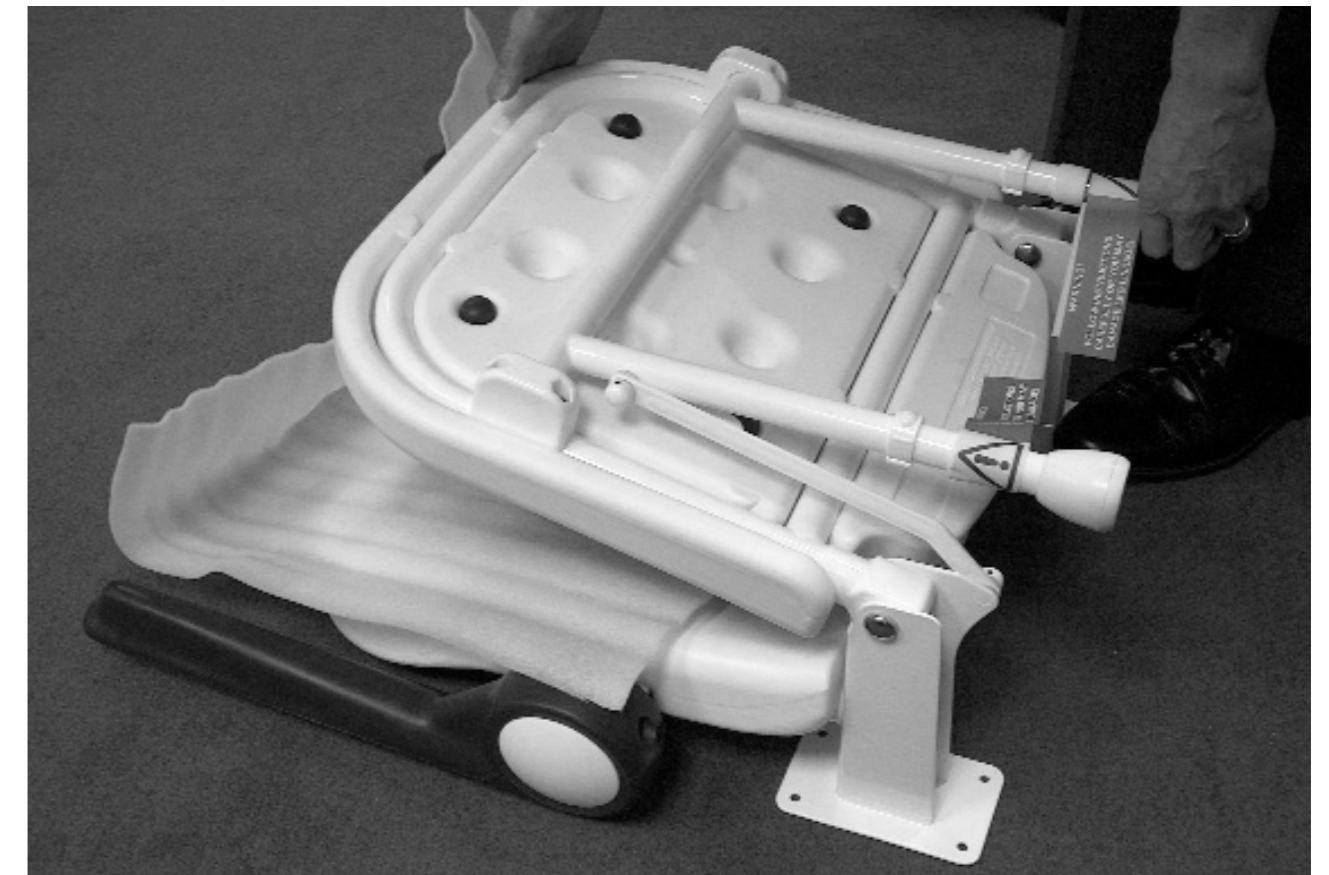

- Always hold the seat and legs as shown in the picture above if opening the seat when inverted.
- Never force the legs apart or open the seat by only lifting the legs.
- Refer to the fitting guidance for size and location of the fixings required.
- After installation check that all 4 of the push-on stainless-steel caps at the wall brackets and 2 plastic pivot caps on the leg linkage are firmly in place.

The AKW 4000 series of shower seats has been independently tested to 25,000 operations of the leg/seat mechanism.



The AKW 4000 series shower seats conform to the requirements of the EU Council directive 93/42/EEC for a class 1 non-invasive transient use device, using the following harmonized European standards in the confirming assessment: BS EN 12182:1999, BS EN 12727:2000

Installer: Please ensure that this information is left with the end user of this product.



ALWAYS read the installation instructions before installing this product.

NEVER exceed the weight capacity of 40 stone/ 560lb/ 254 Kg.



ALWAYS read the end user care instructions.

ALWAYS use hole reducers if cushions not used.



ALWAYS have the seats installed by a trained and experienced installer.



NEVER stand on the shower seat.
These actions may reduce the strength of the shower seat and will invalidate any warranty or weight capacity stated.



BEWARE A pinch hazard may be present during the closing of this product up towards the wall.



DO NOT place matches or cigarettes on or near this product.

The following plastic materials are used in this product:
Polypropylene(PP), Nylon (PA),
Thermoplastic Rubber (TPR)
High Density Polyethylene (HDPE)
Polyurethane Foam (PU)

No natural Rubber or Latex used in this product

The measurements throughout these installation and user care instructions have been converted from metric to its equivalent in imperial, therefore all dimensions are approximate and subject to manufacturing tolerances. AKW MediCare does not make any warranty, expressed or implied, or accept any liability in connection with this information or its use. This information is furnished upon the condition that the person receiving it shall make his/her own tests to determine the suitability thereof for his/her particular purpose.

**For further details on this and other AKW products please contact your distributor
- see details on outer packaging**

