

care and maintenance

The Seat2Go should be regularly checked for signs of damage or wear and tear, including:

- Tears or worn upholstery
- Poor operation or damage to locking clips
- Pulled stitching on upholstery and straps
- Damaged straps

The Seat2Go can be wiped clean with damp cloth and mild detergent or disinfectant.

The estimated service life of the device is 3 years in a single user environment. Service life may be reduced if used in a multi-user environment.

safety precautions

- The Seat2Go should always be used under competent adult supervision and as instructed.
- Always ensure that all straps are correct adjusted. Incorrectly adjusted straps can cause risk of suffocation.
- Always ensure the Seat2Go is secure before use if used with a chair or stroller.
- Always ensure that the Seat2Go is used on a flat surface to reduce the risk of instability
- The Seat2Go should never be used as a seat in a motor vehicle.
- Do not exceed the weight limit
- Do not allow a child to use the Seat2Go if they are too large or too small for the device.
- Note: All recommendations contained herein should be checked with your health professional (such as occupational therapist or physiotherapist) before use

warranty details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

drive

Drive Medical Ltd

Ainley's Industrial Estate, Elland,
West Yorkshire, GB HX5 9JP
www.drivemedical.co.uk

Document code: SEAT2GO_HANGTAG (Issued Nov 2011)

**IMPORTANT: READ ALL INFORMATION
HEREIN BEFORE OPERATION**

Seat2Go

drive



www.drivemedical.co.uk

assembly

Your Seat2Go comes in one carton and is already fully assembled.

use and adjustments

Your Seat2Go can be used as a floor sitter, booster chair and as an insert in a stroller. They are suitable for some children that need support while sitting on the floor at the table or at play. The CRS2000 has a weight limit of 50lb (23kg) and the CRS3000 has a limit of 75kg (34kg).
NOTE - not be used as a seat in a motor vehicle

The straps below (A- D) need to be adjusted to suit each user. The straps should be taut and supportive, but ensure that they are not too tight to restrict the user's breathing or comfort.



Note: All recommendations contained herein should be checked with your health professional (such as occupational therapist or physiotherapist) before use

to use as a floor sitter

1. Place the Seat2Go on a flat surface.
2. Open the locking clip (A) on the pelvic belt. Detach the two shoulder straps (B) from the velcro straps on the Seat2Go back. Open the locking clip (C) on the chest support (D) and pull the support apart.
3. Place the child in the Seat2Go and fasten the pelvic belt (A).
4. Wrap the chest support (D) snugly around the child, and secure using the velcro on the chest support and the locking clip (C).
5. Determine the correct fit for the shoulder straps (B) and attach the velcro strips on the Seat2Go back. The shoulder straps should be supportive and taut but not pulling up on the chest support or uncomfortable for the child.

booster chair and stroller

1. Remove the fastening belts from the pocket on the seat bottom (E) and Seat2Go back (F).
2. Place the Seat2Go on a suitable chair or in a stroller, wrap the belts around for the back and seat of the chair / stroller and secure by closing the locking clips.



3. Ensure that the chair is secured safely in place and that the belts are taut.
4. Insert the child as per the instructions for the floor sitter.



Note: The Seat2Go should not be used as a seat in a motor vehicle.