

User Instructions

Home Helper Trolley

M03792

BROWN

P11917

WHITE

General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 Intended Use

The Home Helper Trolley is ideal for indoor home use. The strong and versatile frame is height adjustable and is fitted with two trays to provide a safer means for users to move items around the home. The lower tray is set forward to allow users to walk naturally when pushing.

2.0 Safety Precautions

Do not use the trolley as a walking aid.

Do not use the trolley to assist when attempting to stand from a seated position.

Do not attempt to sit on the trolley.

Do not overload the trolley trays. When possible it is advisable to position items centrally.

Do not use the trolley outdoors.

3.0 Unpacking

Remove packaging and check all parts carefully before assembling the product (figure 2). Do not assemble if any parts are damaged.

4.0 Assembly

Attach the handles to the Trolley Frame:

- Slot the handles into the top of the trolley frame, figure 3.

Ensure the raised end faces towards the front of the trolley.

- Depress the brass button clip to allow the handle tube to slide inside the trolley frame tube.

Ensure the button clip clicks back into place.



Figure 1 – Model M03792

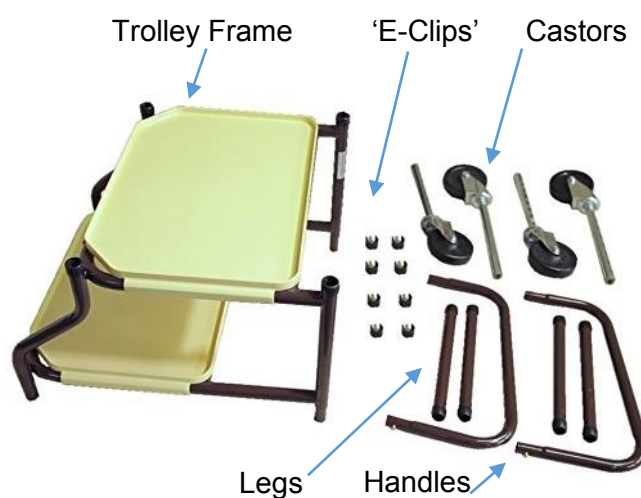


Figure 2 – Contents

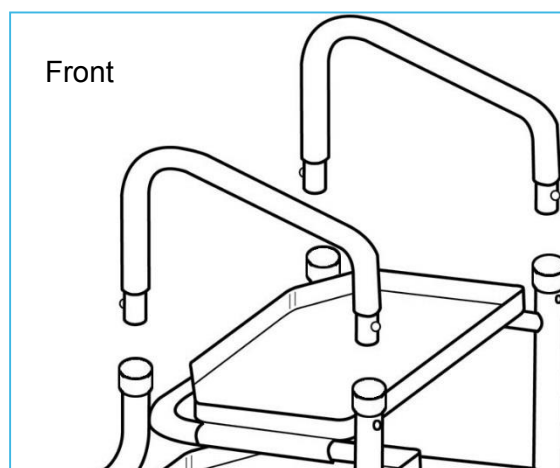


Figure 3 – Attaching the Handles

Attach the legs to the Trolley Frame:

- Slot the leg tube over the ends of the trolley frame, figure 4.
- Align the holes and insert an 'e-clip'.
- Repeat for all 4 legs.

Attach the castors to the legs:

- Slide the castor inside the leg tube.
- Align the holes at the desired height and insert an 'e-clip'.
- Repeat for all 4 castors.

Ensure all 4 castors are set to the same height to prevent instability.

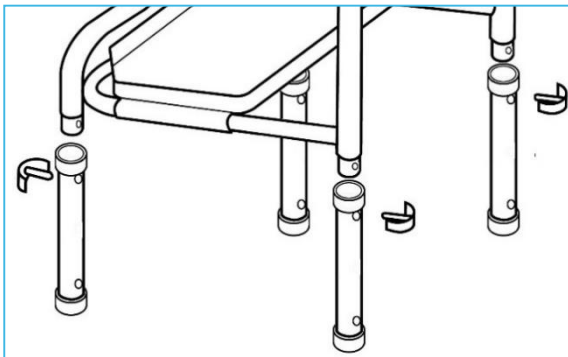


Figure 4 – Attaching the Legs

5.0 Adjustment

To adjust the height:

- Remove the 'e-clip' securing the castor in position.
- Slide to the required height.
- Align the holes and re-insert the 'e-clip'.
- Repeat for all 4 castors.

Ensure all 4 castors are set to the same height to prevent instability.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business park, Bardon Hill,
Coalville, Leicestershire, LE67 1UB, UK

6.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

If required the trays can be removed for cleaning, simply unclip from the frame.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners/cloths as this will damage the surface.

Allow product to dry thoroughly before use.

7.0 Maintenance

Regularly check the condition of the castors and remove any build-up of dirt or debris.

Check that all the fixing clips are inserted correctly and that there is no excessive movement.

Periodically check the trolley for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

8.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

UI/M03792, P11719/10/17/lss5

www.nrshealthcare.co.uk