

User Instructions

Wall Mounted, Folding Shower Chair Range:

**M53370 - Wall Mounted Shower Chair
with legs, arms & back**

**M53540 - Wall Mounted Shower Chair
with Legs, arms, Padded Seat & Back**

**Maximum
User
Weight**



General Safety Warning

Please ensure these instructions are fully read and followed. Failure to do so may result in injury to the user. Retain these instructions in a safe place for future reference.

Intended Use


This wall mounted folding shower seat range, with legs, back and arms is intended to be used in a shower to provide a stable, seated support, where a user is unable to stand for long periods (i.e. during showering). It can be folded up against the wall when not in use.

Unpacking

Remove all parts from packaging, Check all parts are present. Check for damage. If you see, or suspect a fault, do not use the product and contact your supplier.

Safe Installation Information:

- This Shower Seat should be correctly installed by a suitably qualified, competent person.
- Only install the product in an area where the mounting surface is structurally sound. (Strengthening modifications to the fixing surface may be required.)
- This product must be secured to the wall using fasteners which are appropriate to the structural base material.
- DO NOT install in fibreglass / acrylic shower enclosures with insufficient stiffness and strength.
- The maximum user weight for this product is 160kg (25st) based on the product being fitted to a suitable surface using appropriate fixings and fitted by a competent person.
- Waterproofing is recommended, and all fixings used should be non rusting. Sealant should be applied around wall contact points to avoid water ingress.
- Inspect the product at regular intervals for wear and damage, do not use if unstable or the rubber feet are damaged

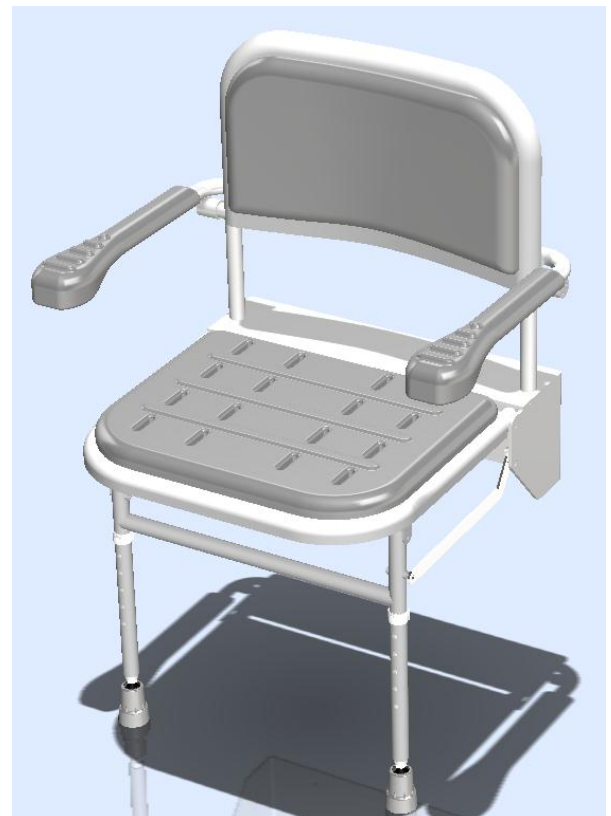

M53540 shown

Fitting and Height Adjustment

1. Identify the correct height and position for the seat within the shower. **IMPORTANT;** When choosing the position, check there are no concealed electrical cables or pipe work near to where the fixing holes are to be drilled.
2. Using the e-clips and external leg holes, adjust the leg extension tubes to the correct height for the client. For finer adjustment the legs may be adjusted individually by unscrewing the threaded feet and locking into position by tightening the nut
3. With the seat in the down position, 90 degrees to the wall, and the legs vertical, with the feet flat and secure to the shower base, mark the fixing hole positions for the wall mounting bracket on the wall.
4. Using appropriate, non rusting screws and fixings, secure the mounting bracket to the wall.
5. Check the seat can be raised and lowered without interference, and that the feet are securely in contact with the shower base.
6. Check seat is firm and secure.

Using the Shower Stool

- Before using, test with both hands, to check the seat is securely fastened to the wall
- To lower; pull down on the seat edge,
- Ensure both legs are level and in contact with the shower floor before sitting.
- Always sit centrally on the shower seat.
- The arms can be raised or lowered individually to allow easier side transfers
- After use, you may lift the seat back up against the wall.
- Do not stand on the seat.
- Do not lower, or raise, the seat using the legs, as this may cause damage.

Care and Use

- Inspect the seat at regular intervals, ensure the product is stable and rigid.
- Check all fixings are fully tightened and in good condition.
- Check ferrules monthly for signs of wear.
- The frame & seat may be cleaned using a non-abrasive cream cleaner or detergent solution.

Other products you may find useful from our range of bathroom accessories

Corner Shelf - Code:M66376



Soap Dish - Code:M66388



Long Handled Sponge - Code: F22594



Long Handled Toe Washer - Code: L63630



Additional copies of this user instruction are available on our website: www.nrs-uk.co.uk.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business Park, Bardon Hill,
Coalville, Leicestershire.
LE67 1UB UK

Tel. Main Office: 0330 100 00 98
Tel. Customer Services: 0845 120 4522
Email: customerservice@nrs-uk.co.uk

These products are from a range of over 3000 daily living aids available from NRS Healthcare. To view the full range visit : www.nrs-uk.co.uk or request a catalogue from the above number.

www.nrs-uk.co.uk