

User Instructions

Economy Shower Stool

L97718

Maximum User Weight:



General Safety Warning

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

Intended Use

The Economy Shower stool is height adjustable and is intended to be used in a shower to provide a stable, seated support, where a user is unable to stand for long periods (ie. during showering).

Specification

Seat height Adjustable from 35.6-53.4cm
 Seat Size 32.5cm diameter
 Footprint 38.5 x 38.5cm
 Max User Weight: 136 Kg (21st)

Contents	Quantity
Bottom Leg 1	1
Bottom Leg 2	1
Long Screw	5
Cupped Washer	5
Seat	1

Assembly and Height Adjustment

Tools required: Philips Screwdriver

1. Put seat upside down on floor/flat surface
2. Align seat legs on the seat according to the number indicated on the legs and seat.
3. Install mounting screws through the cupped washer, top legs bottom leg and tighten to the seat. Check seat is secure and stable.
4. Adjust to the correct height by depressing the snap buttons and sliding the leg extensions up/down the frame tube until it snaps into the appropriate adjustment hole.
5. Ensure all four leg extensions are secured to the same adjustment hole to ensure even height.



Using the Shower Stool

Place in shower. Ensure all four legs are level and in contact with the shower floor. Sit centrally on the shower stool.

Safety Precautions

Do not exceed the maximum user weight limit. Inspect at regular intervals for wear and damage. Do not use if rubber feet are damaged.

Cleaning Instructions

Clean regularly, wiping down with mild detergent /disinfectant solution.

Manufactured in China for

Nottingham Rehab Supplies

Clinitron House, Excelsior Road, Ashby Park
 Ashby de la Zouch, Leicestershire, LE65 1NG
 United Kingdom

Tel: 0845 120 4522

Int'l: +44 (0) 1530 418378

Fax: 0845 121 8112

Email: customerservice@nrs-uk.co.uk

This product is from a range of over 3000 daily living aids available from Nottingham Rehab Supplies. To view the full range visit www.nrs-uk.co.uk or request a catalogue from the above number.