

# USER INSTRUCTIONS

## SPA PERCHING STOOL

P42409

Spa Perching Stool

### ⚠ GENERAL SAFETY WARNING ⚠

Please make sure that you read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

### INTENDED USE

The SPA Perching Stool has been designed to provide users with a stable seated support for use during showering. The Perching Stool is suitable for users with reduced mobility, who are unable to stand up for long periods of time. The sloped seat design provides a relaxed perching position which also makes it easy to rise when standing.



### SAFETY

The maximum user weight must not be greater than 20st / 127kg.

Do not stand on the stool under any circumstances, failure to comply can result in serious injury.

Do not lean backwards on the stool, it can become unstable and fall.

Make sure that all legs are set at equal height and in contact with the floor, to avoid rocking and instability. Do not use in showers.

### MAXIMUM USER WEIGHT



### UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

### Contents


1. SPA Perching Stool

### ASSEMBLY

The perching stool is supplied pre-assembled, at the lowest height setting.

#### 1.0 Adjusting the Height

**Note:** The stool must be set at a height that allows the user's feet to be fully in contact with the floor at all times.

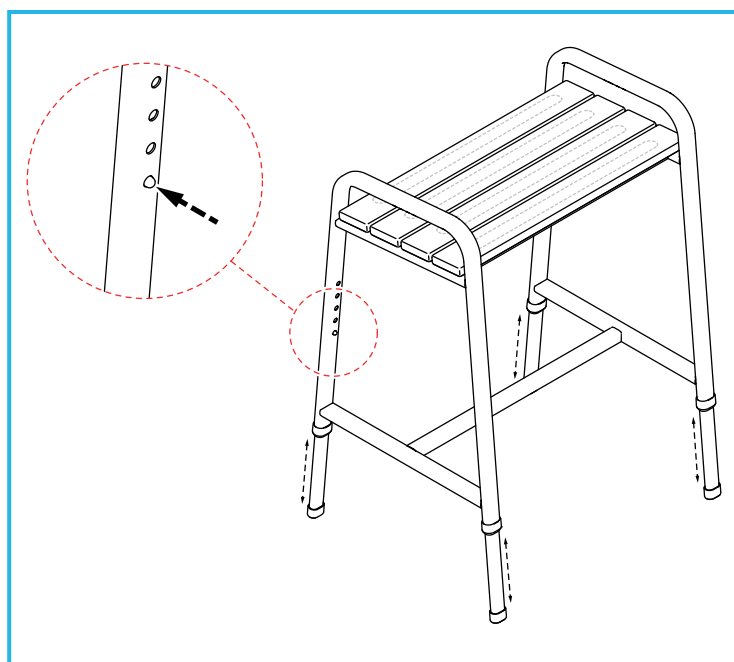

Push and hold the leg button-clips, then slide the legs to the required height until the button-clips click into position in the holes.

**Note:** Make sure that the button clips have fully engaged the holes.

**Note:** Make sure that all legs have been adjusted to the same height before use.

### CLEANING

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry fully prior to use/storage.

### MAINTENANCE

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules and non-slip strips.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

### ACCESSORIES AND SPARES

P57875: Replacement set of ferrules.

P57851: Replacement set of 10 non-slip strips.

### PRODUCT DISPOSAL

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal, plastic and bamboo and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

### PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

## OTHER NRS PRODUCT IN THE SPA RANGE

### P23075 - SPA Shower Stool



### P23099 - SPA Bath Step



**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel. Customer Services: 0345 121 8111

Email: [customerservices@nrshealthcare.co.uk](mailto:customerservices@nrshealthcare.co.uk)

NRS Healthcare is a leading supplier of community equipment and daily living aids, to view our full range, visit our website:

[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)

P23075-UI-2