

User Instructions

Standard Folding Support Rail with Leg M11259

Maximum User Weight



General Safety Warning

This product should only be fitted by a suitably qualified person.

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

Intended Use

The Standard Support Rail with leg is a high quality wall mounted steel rail which provides a stable support to assist the user raise and lower themselves for toileting. The rail can be raised back to the wall when not in use or when transferring from the side. It is not intended for use in a shower.

Safety Precautions

This device should only be secured to a solid wall of brick or concrete construction and be installed by a suitably qualified person.

Test the rail after installation to ensure that it is securely fixed to the wall.

Periodically check the stability and rigidity of the product and, where applicable, that all screws are tightened and in good condition.

Do not exceed maximum user weight.

Installation Instructions

1. Ensure that when correctly located, the rail can be raised and lowered without interference from other fixtures and furniture and that transfer can be carried out safely.
2. Using the pin clip, adjust the leg length to the required position/ rail height.
3. Position the Wall Plate against the wall at the height required (with leg square to and in contact with the floor) and mark the position of the 6 holes.
3. Drill 6 holes, as appropriate for the fixings/screws being used. Insert wall plugs and gently tap them home.



4. Place Wall plate against wall and locate suitable screws into the holes. Ensure screws are screwed firmly to the wall.

5. When fitted, check the product is securely fixed to the wall and that the arm can be raised and lowered without interference, and that the leg is in contact with the floor when lowered. Also check that the friction lock is effective and that the support rail will hold itself at any angle during movement (i.e. it does not drop down on it's own).

Cleaning Instructions

Clean all parts regularly, wiping down with mild detergent /disinfectant solution.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business Park, Bardon Hill,
Coalville, Leicestershire,
LE67 1UB,
United Kingdom.

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0845 121 8111

Email: customerservice@nrshealthcare.co.uk

This product is from a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range visit:

www.nrshealthcare.co.uk or request a catalogue from the above number.