

Product Technical Data Sheet 019

Nuvo Corner Shower Stool

Product Range

N05370	Nuvo Corner Shower Stool—Standard Seat
N05382	Nuvo Corner Shower Stool— Padded Seat

Product Description & Intended Purpose

With a frame made from coated steel tube and a polypropylene seat, this corner shower stool is intended to be used in a shower to provide a stable, seated support, where a user is unable to stand for long periods (i.e. during showering).

Product Features

- Fits neatly in the corner of a shower cubicle
- Height Adjustable
- Seat Drainage Holes
- Anti slip Textured Seat
- Concealed Adjustment holes



Technical Specification

Dimensions (mm)

- Seat Height Min: 450
- Seat Height Max: 600
- Seat Depth: 304
- Seat Width: 405
- Footprint 560 wide x 390 deep

Max User Weight: 127kg (20st)

Installation Information *(Note: for detailed information refer to User Instructions)*

- The shower stool is delivered fully assembled.
- To adjust height, remove 'e-clips' and slide inner tube to desired position, align the holes and re-insert 'e-clip'.

Safety Precautions

- Do not exceed maximum user weight of 127kg (20st).
- Avoid extending the legs in a manner which causes the stool to slant. Care should be taken when leaning forward on the stool.
- Do not attempt to climb or stand on the stool.
- Do not move the stool whilst occupied.

Product Technical Data Sheet 019

Nuvo Corner Shower Stool

Cleaning

- To clean the seat use a neutral detergent and warm water, wipe seat and dry with a disposable paper towel.
- To clean the steel frame, detach the seat and use a neutral detergent and warm water to thoroughly wipe the tubing.
- To disinfect wipe with a dilute bleach solution and rinse thoroughly with warm water.
- Please check the suitability of the detergent prior to use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Always allow the product to fully dry prior to stacking and storage.

Care & Maintenance

- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.
- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Check for general wear or damage. Any damage should immediately be reported to the supplier and the product taken out of service.

Accessories / Related Products

None

FAQ's

- Will the stool be stable on wet floors?
The product is fitted with rubber ferrules to provide greater grip and stability on all surface.
- When using the shower stool will I be sitting in a pool of water?
No, the seat has several holes in it to allow water to drain through.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardonia Hill, Coalville, Leicestershire. LE67 1UB UK