

USER INSTRUCTIONS

EXTRA WIDE PERCHING STOOL

L96751

Extra Wide Perching Stool

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

To be used to provide a stable, seated support, where a user is unable to stand for long periods. The seat slopes forward to allow the user to achieve a relaxed “perching” position when sitting and also to assist when rising. This product is not for use in a shower or wet-room.



2. SAFETY

Warning: The maximum user weight must not be more than 254 kg (40 st).

Inspect product at regular intervals for signs of wear or damage.

Make sure the floor surface is level and even to avoid rocking.

Do not use if you suspect a fault.

Do not use this product in a shower or wet-room.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. IN USE

4.1 Stool Height Adjustment

Remove the plastic e-clips connecting the stool legs and leg extensions.

Slide the leg extensions to the required position.

Note: Make sure that all legs have been set to the same height.

Replace the plastic e-clips making sure that each e-clip is fully connected.

5. MAINTENANCE

Check that all fixing e-clips are inserted correctly.

Check for general wear or damage.

If you see or suspect a fault do not use – contact your dealer immediately.

6. CLEANING INSTRUCTIONS.

Clean regularly, use a general purpose, neutral detergent and warm water.

Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.

Dry thoroughly before use.

7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk