

USER INSTRUCTIONS

WOODEN TRANSFER BOARD RANGE

F20421	Short Transfer Board (600mm)	F20433	Long Transfer Board (760mm)
F20998	Curved Transfer Board	M01175	Lightweight Curved Board

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

These Transfer Boards are designed to assist users transferring between two seated surfaces of similar height eg. wheelchair to bed.

2. MATERIAL SPECIFICATION

Birch Plywood with clear lacquer finish.



3. SAFETY

WARNING: The maximum user weight must not be more than 190Kg (30st).

DO NOT: Attempt to transfer across distances greater than the board allows. Ensure both ends of the board sufficiently overlap the transfer surfaces. NRS Healthcare recommend a minimum of 150mm.

DO NOT: Attempt to use multiple boards for a single transfer.

Transfer surfaces should be as level as possible, avoid slopes.

Maximum User Weight



4. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

5. IN USE

Position the transfer surfaces next to each other, at right angles as close as possible.



Figure 1: M01175 in use example

Place the transfer board over the gap between the two surfaces (refer to figure 1). Ensure it is sufficiently supported on both sides, a minimum of 150mm overlap.

The user can now use the board to manoeuvre from one surface to the other.

Note: Take care to ensure fingers do not become trapped under the board.

Where possible the user should push down with their hands during transfer to lift themselves slightly to avoid rubbing on the board. This is particularly important where there is a loss of sensation. Assistance from a carer may be required in cases of severe limitation.

6. MAINTENANCE

Inspect the product regularly for any signs of wear or damage to the surface of the board which may cause friction and damage the skin.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

7. CLEANING INSTRUCTIONS.

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from wood composite and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk