


- M11120 - 2" (50mm) Stanton Raised Toilet Seat with Lid
- M11132 - 4" (100mm) Stanton Raised Toilet Seat with Lid
- N05266 - 6" (150mm) Stanton Raised Toilet Seat
- N05289 - 6" (150mm) Stanton Raised Toilet Seat with Lid

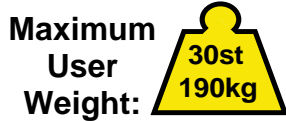

### General Safety Warning

**Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.**

#### 1.0 Intended Use

The Stanton Raised Toilet Seat is designed to fit a standard toilet bowl and provide a secure raised toilet seat surface for those with reduced mobility.

#### 2.0 Safety Precautions

- Do not exceed the maximum user weight limit of 30st (190kg)
- Inspect the Toilet Seat regularly to ensure it is fitted correctly, is safe and secure, and fit for use. If in any doubt, contact a competent installer of sanitary equipment.
- Inspect at regular intervals for wear and damage.
- Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

#### 3.0 Unpacking

Check all parts carefully before assembling the product. Do not assemble if any of the parts are damaged. Do not assemble if any of the contents are missing.

#### Contents


- Raised Toilet Seat
- 2 Part Side Brackets (x2)



### 4.0 Fitting Instructions

#### 4.1 Bracket Assembly

1. Assemble the side brackets; clip the screw (with knob) into "L" shaped bracket as shown in fig 1.

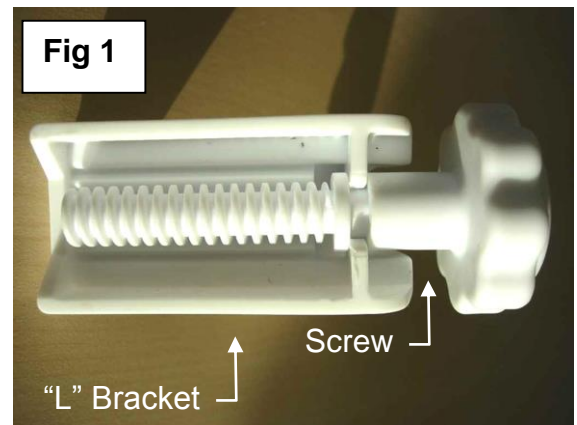

2. Slot the bracket into threaded slots in underside of seat as shown in fig 2 and screw until the thread is fully engaged.



## 4.2 Fitting to the Toilet

1. Check the toilet is safe and secure before attempting to fit the product
2. Lift up the existing toilet seat.
3. Take the raised toilet seat, check the two side brackets are screwed out an equal amount and there is enough clearance to fit over the toilet bowl.
4. Place on the toilet, ensure the front lip is outside of the bowl and the seat is evenly positioned about the bowl (see main picture)
5. **Secure in place by turning the screws anti-clockwise. Assembly tip: To ensure the toilet seat is positioned correctly tighten both side brackets simultaneously by turning both screws, an equal amount, until the seat is secure.**
6. Before use check the seat is secure by applying reasonable force, front to back and side to side, if there is any movement re-tighten the screws.
7. To remove the seat, simply unscrew the side brackets an equal amount and lift clear.

## 5.0 Correct Use

- We recommend that you regularly check the seat, ensuring that the screws are tight and the seat remains evenly positioned on the bowl.
- If you suspect that the seat has moved out of position, please follow the instructions above to move it back into the correct position and retighten all fixings.
- Do not under any circumstances stand on the seat or allow others to do so as this is highly dangerous and will compromise the safety of the user.
- If you have to remove the Raised Toilet Seat for any reason, please ensure that you follow the installation instructions again fully before using. Failure to do so could compromise the safety of the user.



## 6.0 Cleaning Instructions

- Clean regularly, use a general purpose, neutral detergent and warm water
- Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water
- These seats may be autoclaved on cleaning cycles up to 73°C
- Dry the seat thoroughly before use.

## 7.0 Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

## 8.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product. Any guarantee issued by NRS Healthcare does not affect your statutory rights.

## 9.0 Contact Details

**NRS Healthcare,**  
Sherwood House, Cartwright Way,  
Forest Business Park, Bardon Hill,  
Coalville, Leicestershire,  
LE67 1UB  
United Kingdom.

Tel. Head Office: 0330 100 0098  
Tel. Customer Services: 0345 121 8111  
Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:  
[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)