



## 2 Easy Steps to Independence

### 1 Sitting Down

**\*LOCK THE WHEELCHAIR BRAKES BEFORE SITTING DOWN!**

- Sit Comfortably in the center of the chair.
- Start deflating by pressing the **Down** arrow button.
- You may now sit down comfortably.

#### \* NOTES

- One click will deflate one chamber.
- In case of cognitive impediment the SitnStand should only be operated by a caregiver.




### 2 Standing Up

**\* LOCK THE WHEELCHAIR BRAKES BEFORE STANDING UP!**


- Take your feet off the wheelchair pedals and place them on the floor, close to the chair.
- Sit upright.
- Spread your feet.
- Inflate the cushion by pressing the **Up** button.
- Now you can easily stand up.

#### \* NOTES

- One click will inflate one chamber.
- Do not move the wheelchair until the cushion is fully deflated.

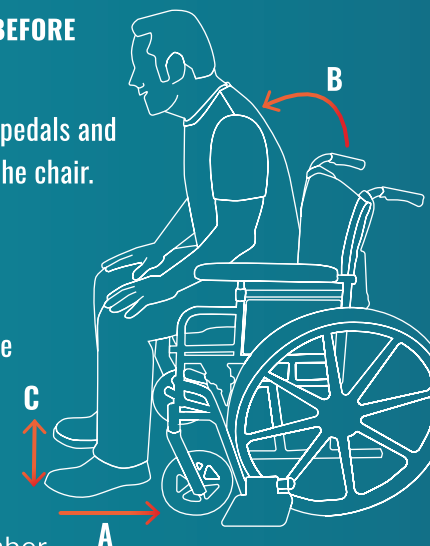

### i Get to know the Remote control

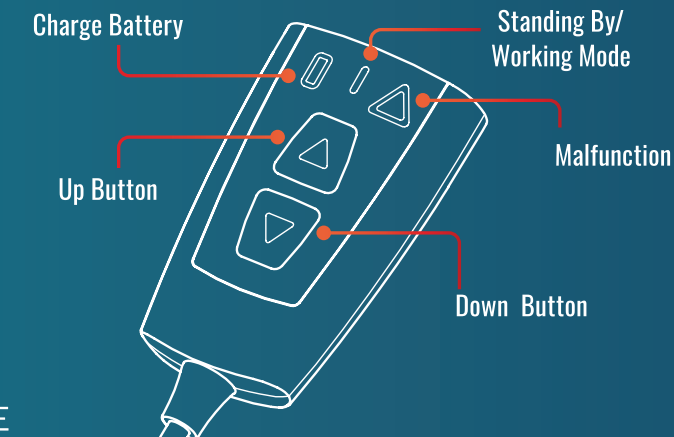

#### \* NOTE

Standby mode - flashes slowly (one flash/sec).  
Working mode - flashes quickly (5 flashes/sec).  
Malfunction reset - press the two control buttons simultaneously for 10 secs.  
If that doesn't help, please contact us at – [info@sitnstand.com](mailto:info@sitnstand.com)

# Placing the SitnStand

**\*LOCK THE WHEELCHAIR BRAKES BEFORE PLACING THE SITNSTAND!**

A. Place the folded SitnStand on the wheelchair seat - **Illustration A.**

B. push the air supply unit through the gap at the rear of the seat  
**Illustration B.**

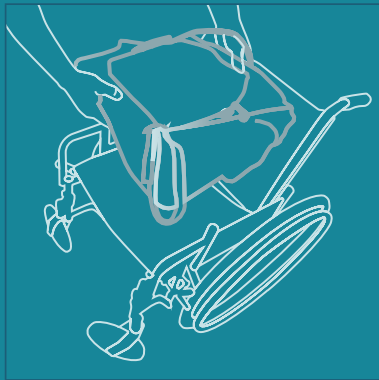


Illustration A



Illustration B



Use this velcro to attach the controller to wheelchair as detailed below

C. Place the straps of the air supply unit on the handles of the wheelchair and fasten them tightly - **Illustration C.**

D. When not in use we recommend you attach the remote control to a velcro strip that will be attached to the wheelchair armrest  
**Illustrations D1 + D2.**

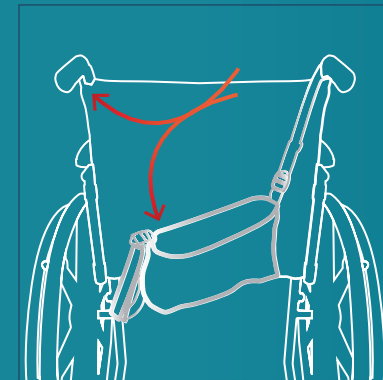


Illustration C

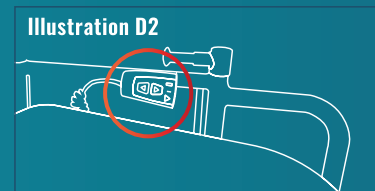
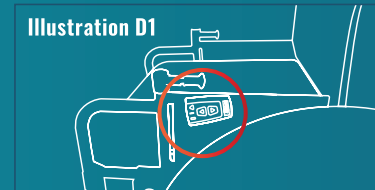


Illustration D1

Illustration D2


# Charging the Battery

A. Remove battery by sliding it towards you.

B. Connect battery to the charger as illustrated.


C. When the **Red** light on the charger changes to **Green**, the battery is fully charged.

D. Slide the charged battery into its place.

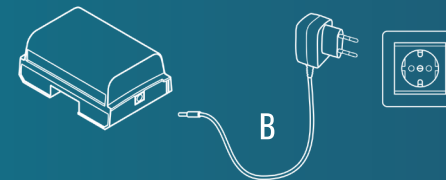

## \* NOTE

When using the battery for the first time, it should be charged for at least 12 hours.

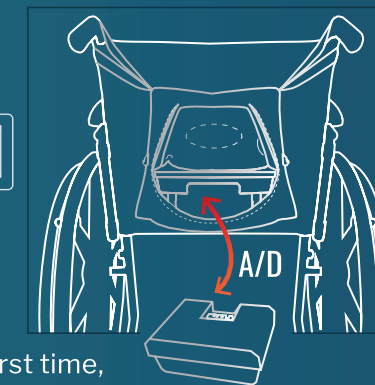

# Keep This Card

**Congratulations for buying the SitnStand for wheelchairs!**

**Scan to watch Step-by-Step tutorial videos**



We hope that the SitnStand for wheelchairs will offer you greater independence for many years, and allow you to move freely wherever you go.

The SitnStand is a quality product. We have invested time and effort to ensure it is delivered to you in prime condition to serve you for many years. If for any reason you experience any discomfort during use, please do not hesitate to contact us.

**SitnStand**  
For Wheelchairs

✉ email [info@sitnstand.com](mailto:info@sitnstand.com)

🌐 web [www.sitnstand.com](http://www.sitnstand.com)