

Product Technical Data Sheet 031

NRS Handling Belt

Product Range

L98462	Paediatric	L98498	Large
L98474	Small	M09496	MAXI
L98486	Medium	M09502	MINI

Product Description & Intended Purpose:

The handling belts help carers to stabilise a client when rising to a standing position and to assist when walking without needing to grip his/her body.



Product Features

- Contoured design to better fit the body.
- Fleece lining complete with anti-slip section for added comfort and security.
- Quick release buckle.
- Padded handles for extra comfort grip for carer.
- Hanging loop for storage.
- Machine washable.

Technical Specification:

Code	Size	Waist Size
L98462	Paediatric	410-740mm (16-29")
L98474	Small	510-790mm (20-31")
L98486	Medium	787-1120mm (31-44")
L98498	Large	1092-1422mm (43-56")
M09496	*MAXI	610-1140mm (24-45")
M09502	*MINI	510-1140mm (20-45")

Safety Precautions

- Persons using this Handling Belt should have received appropriate training or instruction.
- Transfers should only be attempted following a comprehensive risk assessment and should be performed in accordance with local Moving & Handling Guidelines and the current edition of the Guide to The Handling of People (hop).
- Do Not use the Handling Belt to lift a client.

PTDS031 NRS Handling Belt

Cleaning

- Machine wash warm at 70°C for 30 minutes.
- Do not use bleach, biological detergents or detergents containing bleach.
- Do not spot clean with strong chemicals.
- Do not iron.
- Do not dry clean.
- Do not tumble dry.
- Air dry thoroughly prior to use or storage.

Care & Maintenance

- Periodically check the product for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of service.

Accessories

None

FAQ's

XXX

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK