

USER INSTRUCTIONS

CHILDREN'S COMMODE

M46419

Children's Commode

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

The Children's Commode is designed to assist children who have difficulty in accessing or using a standard toilet. It has a backrest, armrests and also a Lap Belt to give additional support if required.



2. SAFETY

Warning: The maximum user weight must not be more than 70Kg (11st).

Before using the commode, check the product for any visible defects. If you see any damage or suspect a fault, please contact your supplier for further instructions.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

Maximum User Weight



4. INSTRUCTIONS FOR USE

Adjust the height of the commode, to suit the child, as follows:

1. Remove each 'E'clip in turn. Slide each leg to the required height.
2. Align the holes and replace 'E' clip.
3. Ensure that all legs are secured at the same position in order to prevent rocking.

Note: Never extend legs in a manner so that the commode is slanting. This will compromise the safety of the user.

4. Adjust the Lap Belt to suit the child, if required.
5. To remove the Potty, unclip and raise seat at the front.
6. For cleaning instructions, refer to Section 5.

Spare Commode Potty and Lid:

The potty and lid are available as a spare. To order please contact customer services and quote Code No. M11193.

Please Note: Under no circumstances should the commode be moved whilst occupied. It is advised that you check the frame of the product annually for signs of wear and tear, dimpled or flaking paint or cracks in the frame should be reported to your supplier immediately. Should there be any such evidence, please refrain from using the commode.

5. CLEANING INSTRUCTIONS.

These instructions are intended to provide guidance on the methods and products available for the effective cleaning of the Commode when used in the home; to help reduce the risk of infections as well as keeping the Commode clean and odour free.

It is recommended that you have the following items available (these are widely available but can also be purchased from NRS).

- i) Vinyl or latex disposable gloves
- ii) A sanitising spray
- iii) Sanitising wipes

A) - Cleaning the Commode Pan:

1. Put on the disposable gloves.
2. Place the lid on the potty and carry with both hands to the bathroom or toilet. (It is wise practice to support the potty with one hand in case the handle detaches). Lift up the toilet seat and pour the contents into the toilet.
3. Rinse the potty with some warm water and pour down the toilet; this helps to clean some of the residue left in the potty.
4. Wipe the inside of the potty with sanitising wipes and dispose of these in a separate container.
5. Rinse the potty with warm water and wipe dry with toilet paper.
6. Spray the potty with the sanitising spray and return under the commode seat ensuring it is secure.
7. Remove and dispose of gloves.

B) - Cleaning the Commode

It is recommended that the seat and arms are wiped with sanitising wipes after each use/day, and the whole commode is wiped down once a week.

C) - Cleaning the Lap Strap

The lap strap may be machine washed at temperatures up to 80°C.

Remove the straps from the commode frame, clip together, and ideally place in a net wash bag. For effective decontamination wash at a minimum of 70°C for 10 minutes.

6. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

7. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk